medi USA, LP
6481 Franz Warner Pkwy.
Whitsett, NC 27377
USA
www.mediusa.com

medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth, Germany
www.medi.de

83 Fennell Street
North Parramatta NSW 2151
Australia
T +61 2 9890 8696
F +61 2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43-512 57 95 15
F +43-512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T: +32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Itapeva 538 cj. 42
Bela Vista, São Paulo - SP
CEP: 01332-000
Brasil
T +55 (11) 3500 8000
sac@medibrasil.com
www.medibrasil.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbaek Strand
Denmark
T +45 70 25 56 10
F +45 70 25 56 20
kundeservice@sw.dk
www.meditandmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi.hu
www.medi.hu

medi Italia s.r.l.
Via Giuseppe Ghedini, 2
40069 Zola Predosa (BO)
Italy
T: +39-051 6 13 24 84
F: +39-051 6 13 29 56
info@medi-italia.it
www.medi-italia.it

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Norway AS
P.B. 238
4291 Kopervik
Norway
T: +47 52 84 45 00
F: 47 52 84 45 09
post@medinorway.no
www.medinorway.no

4 049772 412228 E011786/09.2017

medi

CIRCAID

circaid® juxtacures®

Inelastic compression system

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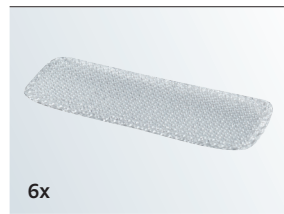
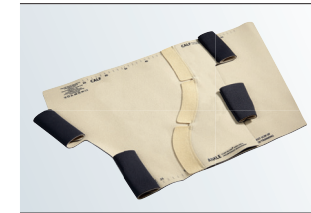
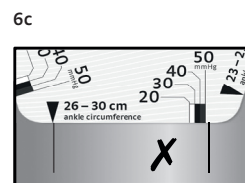
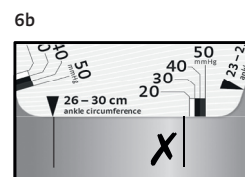
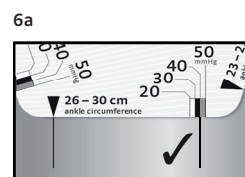
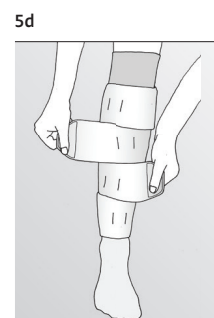
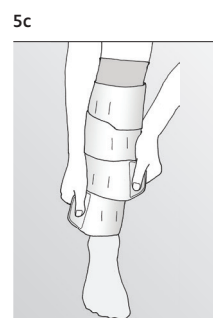
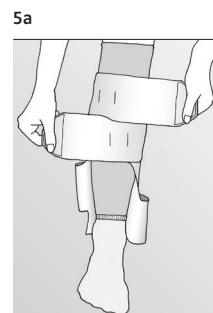
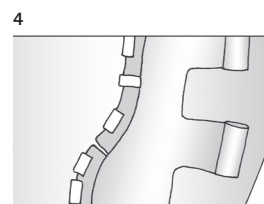
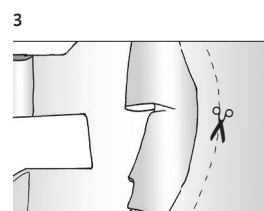
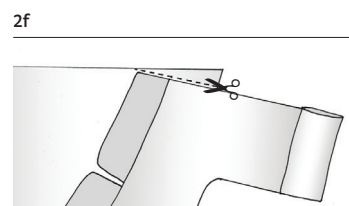
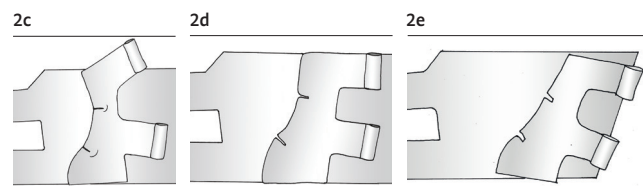
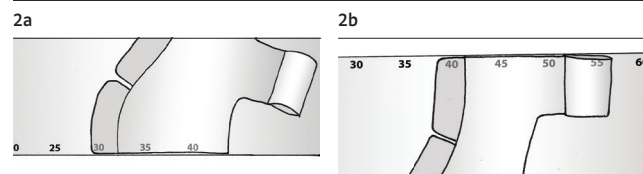
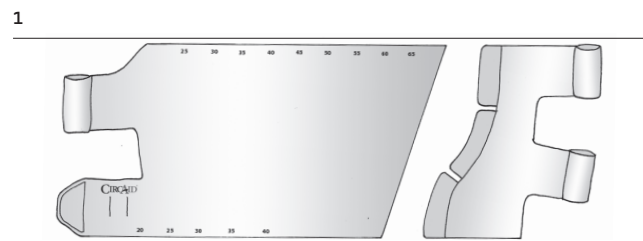
circaid® juxtacures®

Purpose

This device is designed to provide lower leg compression to patients with venous disorders.

The device consists of a series of juxtapositioned, inelastic bands extending from the central portion of the garment. The device contours the leg and is designed to cover from just above the ankle bone to just below the knee.

Indications

- Chronic venous insufficiency
- Venous stasis ulcers
- Post sclerotherapy
- Post thrombotic syndrome
- Dependent oedema
- Varicose veins

Relative contraindication

Due to a known contraindication the attending doctor evaluates the wearing of a medical compression garment (for a suitable indication such as chronic venous insufficiency, etc.), and compares the possible risk with the therapeutic benefit of the medical device.

- Decreased or absent sensation in the leg
- Intolerance to compression material

Absolute contraindication

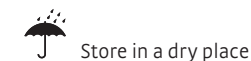
A condition that completely forbids wearing the medical compression garment despite a suitable indication (e.g. chronic venous insufficiency, etc.).

- Severe or moderate peripheral arterial disease
- Decompensated congestive heart failure
- Septic phlebitis
- Phlegmasia cerula dolens
- Untreated infections of the limb

This Package contains

- 1 circaid juxtacures
- 6 Velcro Stays
- 2 circaid undersleeves lower leg
- 2 circaid compression anklets
- 1 built-in pressure system guide card
- 1 Paper Measuring Tape
- 1 Booklet Instructions for Use

Storage instructions

Store in a dry place

Keep out of sunlight

Lifespan

The Circaid juxtacures has a lifespan of 6 months of daily wear.

Due to material wear-and-tear the medical efficacy can only be expected for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, application, and removal).

medi Orient M.E.A. ZCC
TAURUS GROUP
Suite No. M01 Wasel Al Mamzar Bldg.
Al Wuhida Road, Al Mamzar
P.O.BOX NO. 91464
DUBAI, UAE
United Arab Emirates
T: +971-4 2557344
F: +971- 4 2557399
md@taurusgroup.net

medi Polska Sp. z o.o.
Zygmunt Starego 26
44-100 Gliwice
Poland
T +48-32 230 60 21
F +48-32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal Lda.
Rua do Centro Cultural, no. 43
1700-106 Lisbon
Portugal
T +351 21 843 71 60
F +351 21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
Business Center NEO GEO
Butlerova st., 17
117342 Moscow
Russia
T +7 (495) 374-04-56
F +7 (495) 374-04-56
info@medirus.ru
www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
Unit 1806, Floor 18, Sandhill Plaza No. 1
2290 Zuchongzhi Road
200131 Shanghai
People's Republic of China
T: +86-1366 1668 687
F: +86-2161 7696 07
infocn@medi.cn

medi Sweden AB
Box 6034
192 06 Sollentuna
Sweden
T +46 8 96 97 98
F +46 8 626 68 70
info@medi.se
www.medi.se

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
Great Britain
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Tankova Str. 8, office 35
Business-center „Flora Park“
Kiev 04112
Ukraine
T +380 44 591 11 63
F +380 44 392 73 73
info@medi.ua
http://medi.ua

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000
USA
T +1-336 4 49 44 40
F +1-888 5 70 45 54
info@mediusa.com
www.mediusa.com

Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

Material composition

- Nylon 55%
- Polyurethane 38%
- Elastane 4%
- Polyethylene 3%

This garment does not contain latex.

Washing instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. If doing so, it is recommended that you place the garment in a pillow case to aid in keeping the Velcro lint-free and to protect other clothing from getting damaged. To extend the life of the garment, hand wash and drip dry.

Machine wash warm

Do not bleach

Tumble dry, low heat

Do not iron

Do not dry clean

Do not wring

Wearing recommendation

Always ensure that the correct prescribed pressure range is being applied.

Slightly loosen the bands of the garment for night-time wear. Each vertical BPS line on the bands may be at a different

location on the leg depending on the size and shape of the leg.

If experiencing any pain, or discomfort, reduce the compression (mmHg) applied or remove device. Inform your clinician of your action.

When bands are placed correctly they should slightly overlap, leaving no space between bands.

Do not tuck bands.

The Velcro tabs should never touch the skin!

Selecting the circaid juxtacures length

The circaid juxtacures is available in short length (28cm), standard length (33 cm) and long length (38cm).

The appropriate length is determined by the patient's floor to knee crease measurement. The patient should be measured in standing position.

If the patients measures less than 44 cm, select the short legging. If the patient measures between 44 cm and 49 cm, select the standard legging, and if the patient measures more than 49 cm, select the long legging.

It is important that the length of the juxtacures covers the calf for reinforcement of the calf-muscle pump.

Note:

circaid juxtacures will fit a 19 cm - 42 cm ankle circumference and a 25 cm - 64 cm calf circumference. circaid juxtacures will not accommodate more than a 30 cm difference in circumference from the ankle to the calf measurement. If your patient exceeds these measurements, alternative compression garments are available for order.

The circaid compression anklelet will fit a maximum arch circumference of 36 cm for the Standard size and 41 cm for the Large size, the maximum ankle circumference is 27 cm for the Standard size and 50 cm for the Large size.

If your patient exceeds these measurements, further foot options are available for order.

Customizing the circaid juxtacures:

Step 1: Separate the body **(1a)** and spine **(1b)** from one another, unroll the bottom band and locate the number markings along the bottom and top edge of the body piece **(1a)**.

Step 2: Locate the patient's ankle circumference measurement amongst the printed numbers along the bottom edge. Then, locate the patient's calf circumference measurement amongst the printed numbers along the top edge of the garment.

Step 3: Take the spine **(1b)** and place the bottom Velcro tab on the appropriate number **(2a)**.

Take the top Velcro tab of the spine and place it on the appropriate number **(2b)**. Adjust the center spine tab to create a smooth spine curve **(2d)**.

Note:

There is no one correct spine shape. The garment may not lay flat once the spine has been set **(2c)**. This is to be expected when dealing with more extreme spine angles. If there is a large difference between the two circumferences, the spine may not reach both the bottom and top edges of the body piece. In this case, center the spine **(2e)**. When dealing with more extreme spine angles, excess material sticking out beyond the top of the spine piece can be trimmed **(2f)**.

Step 4:

With the spine in place, fold over the smaller garment piece. Cut the excess material from the body piece leaving approximately 5 cm of overlap **(3)**. Be sure not to cut any other part of the garment.

Note:

If desired, attach the separate Velcro tabs along the spine for additional adhesion **(4)**.

Tip:

You may also trace the spine to easily reattach it in its original position in the unlikely event of the spine becoming detached.

Step 5: Take the BPS-card and highlight the range that contains the patient's ankle circumference. If the patient's measurement falls between two ranges on the card, 30.5 cm for example, choose the smaller range.

Donning the circaid juxtacures:

Step 1: Slide the circaid undersleeve onto the lower leg and over the knee. The cuffed edge of the undersleeve should cover the heel. Ensure that there are no wrinkles in the undersleeve.

Step 2: Align the garment just above the ankle bone and below the knee crease. The black side of the material should face the skin. Unroll the top two bands and loosely secure them to the outside of the garment to anchor the legging in place **(5a)**. This enables you to easily raise or lower the garment for correct positioning.

Note:

The bottom most band can be identified by the circaid-logo on the beige side of the material.

Step 3: Unroll the bottom two bands and secure the bottom most band to the outside of the legging to a firm and comfortable compression level **(5b) (5c)**.

Step 4: While holding the second band, detach the next band. Secure the second band and continue this process with the other bands, going up the leg **(5d)**.

Step 5: The garment should lay flat and wrinkle-free against the leg. Adjust each band to a firm and comfortable compression level. Ensure that the bands are placed in an alternating order.

Step 6: Once all bands are secured, inspect the garment for any gaps or creases. Adjust the bands as necessary. The device should never hurt. If any pain is experienced the pressure can be reduced - if this doesn't provide relief the device can be removed and the physician should be notified.

Note:

Once the correct pressure is set, you may fold the top and bottom part of the circaid undersleeve over the top and bottom edge of the circaid juxtacures garment.

Step 7: Donning the circaid compression ankle

First fold the bottom part of the circaid undersleeve over the bottom edge of the circaid garment. Slide the circaid compression ankle onto the foot and ensure that it is pulled high enough so that it overlaps with the circaid garment. Ensure that there are no wrinkles in the ankle.

Doffing the circaid juxtacures:

To remove the garment, remove the circaid compression ankle first. Then unfold the circaid undersleeve. Detach all bands of the legging starting with the top band. Fold each band onto itself to keep the Velcro tabs lint-free. Remove the garment and then remove the circaid undersleeve.

Using the built-in pressure system guide card:

Step 1: Locate the built-in pressure system guide card in your packaging.

Step 2: Identify the appropriate edge of the card and pressure scale. The edge of the card is determined by ankle circumference. It is recommended that the correct edge and scale be highlighted.

Step 3: Starting with the bottom band, line up the vertical line on the card with one of the BPS lines on the bottom band.

Step 4: Note where the second BPS line lines up with the card's compression ranges **(6a)**.

Step 5: If the BPS line on the garment either falls short **(6b)** or goes beyond the correct compression range **(6c)**, readjust the band as necessary so that the second BPS line on the garment is aligned with the prescribed compression range **(6a)**. Completely loosen the band before reapplying it if too much tension has been applied.

Step 6: Repeat the steps 3 - 5 for each band going up the leg. Be sure to adjust all bands to the same tension to ensure graduated compression. Bands may need adjusting throughout the day to maintain the prescribed compression range

Follow-up visits

Remind the patient to bring the entire circaid juxtacures

system to every follow-up appointment for proper adjustment. At every follow-up visit with the patient, the circaid juxtacures should be checked for necessary adjustments.

Reduction of the leg circumferences:

Should the patient reduce significantly, ankle and calf circumference measurements can be re-taken and the garment can be adjusted to the patient's new size by resetting the spine per steps 1 - 5.

Once the device has been properly adjusted, re-apply the legging to the patient's limb. Be sure to also check whether or not the patient should be using a different edge of the BPS-card as the appropriate range on the card may have changed. Make the appropriate adjustments, if necessary.

Tip

Once the patient's ulcer has healed, circaid juxtalite and mediven round-knitted stockings are available for order to prevent ulcer recurrence.

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