medi USA, LP  
6481 Franz Warner Pkwy.  
Whitsett, NC 27377 USA  
www.mediusa.com

medi Australia Pty Ltd  
83 Fennell Street  
North Parramatta NSW 2151  
Australia  
T +61 2 9890 8696  
F +61 2 9890 8439  
sales@mediaustralia.com.au  
www.mediaustralia.com.au

medi Austria GmbH  
Adamgasse 16/7  
6020 Innsbruck  
Austria  
T +43-512 5795 15  
F +43-512 5795 15 45  
vertrieb@medi-austria.at  
www.mediaustria.at

medi Belgium bvba  
Posthoornstraat 13/1  
3582 Koersel  
Belgium  
T: +32-11 24 25 60  
F: +32-11 24 25 64  
info@medibelgium.be  
www.medibelgium.be

medi GmbH & Co. KG  
Medicusstraße 1  
95448 Bayreuth, Germany  
www.medi.de

medi Brasil  
Rua Itapeva 538 cj. 42  
Bela Vista, São Paulo - SP  
CEP: 01332-000  
Brasil  
T +55 (11) 3500 8000  
sac@medibrasil.com  
www.medibrasil.com

medi Danmark ApS  
Vejlegardsvej 59  
2665 Vallensbaek Strand  
Denmark  
T +45 70 25 56 10  
F +45 70 25 56 20  
kundeservice@sw.dk  
www.medit danmark.dk

medi Bayreuth España SL  
C/Canigó 2 – 6 bajos  
Hospitalet de Llobregat  
08901 Barcelona  
Spain  
T +34 93 260 04 00  
F +34 93 260 23 14  
medi@mediespana.com  
www.mediespana.com


4 049772 365630

E012036 / 09.2017

medi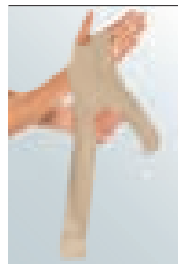

circaid® juxtafit®  
hand wrap

CIRCAID

## circaid® juxtafit® essentials hand wrap

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## circaid® juxtafit® essentials hand wrap

### Purpose

This garment is a single patient, reusable, non-invasive, non-sterile device designed to provide compression to the hand of patients with lymphatic disorders.

### Indications

- Lymphedema

### Relative contraindication

Due to a known contraindication the attending doctor weighs up the wearing of a medical compression garment (for a suitable indication such as Lymphedema, etc.), and compares the possible risk with the therapeutic benefit of the medical device.

- Decreased or absent sensation in the arm
- Allergy to compression materials

### Absolute contraindication

A condition that completely forbids wearing the medical compression garment despite a suitable indication (e.g. Lymphedema, etc.).

- Severe peripheral arterial disease
- Decompensated congestive heart failure
- Septic phlebitis
- Phlegmasia cerula dolens
- Moderate peripheral arterial disease
- Acute infection

### Storage instructions

Store in a dry place

Keep out of sunlight

### Lifespan

Due to material wear-and-tear and the medical efficacy can only be guaranteed for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, donning, and doffing).

medi France  
Z.I. Charles de Gaulle  
25, rue Henri Farman  
93297 Tremblay en France Cedex  
France  
T +33 1 48 61 76 10  
F +33 1 49 63 33 05  
infos@medi-france.com  
www.medi-france.com

medi Hungary Kft.  
Bokor u. 21.  
1037 Budapest  
Hungary  
T +36 1371 0090  
F +36 1371 0091  
info@medi.hu  
www.medi.hu

medi Italia s.r.l.  
Via Giuseppe Ghedini, 2  
40069 Zola Predosa (BO)  
Italy  
T: +39-051 6 13 24 84  
F: +39-051 6 13 29 56  
info@medi-italia.it  
www.medi-italia.it

medi Nederland BV  
Heusing 5  
4817 ZB Breda  
The Netherlands  
T +31 76 57 22 555  
F +31 76 57 22 565  
info@medi.nl  
www.medi.nl

medi Norway AS  
P.B. 238  
4291 Kopervik  
Norway  
T: +47 52 84 45 00  
F: 47 52 84 45 09  
post@medinorway.no  
www.medinorway.no

medi Orient M.E.A. ZCC  
TAURUS GROUP  
Suite No. M01 Wasel Al Mamzar Bldg.  
Al Wuhida Road, Al Mamzar  
P.O.BOX NO. 91464  
DUBAI, UAE  
United Arab Emirates  
T: +971-4 2557344  
F: +971-4 2557399  
md@taurusgroup.net

medi Polska Sp. z o.o.  
Zygmunta Starego 26  
44-100 Gliwice  
Poland  
T +48-32 230 60 21  
F +48-32 202 87 56  
info@medi-polska.pl  
www.medi-polska.pl

## circaid juxtafit essentials hand wrap

< 3 month - depends on the wear and tear

### Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

### Material composition

circaid juxtafit essentials hand wrap  
55% Nylon  
40% Polyurethane  
5% Elastane

This garment does not contain latex.

### Washing instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. To extend the life of the garment, hand wash and drip dry.

Machine wash warm

Do not bleach

Tumble dry, low heat

Do not iron

Do not dry clean

Do not wring

---

### Wearing recommendation

If experiencing any pain, immediately remove the garment. Do not tuck bands. The Velcro tabs should never touch the skin!

### Donning circaid® juxtafit® essentials hand wrap

Step 1: Slide the thumb through the hand wrap's thumb hole. The seam should run from the base of the thumb to the wrist. The curved portion of the wrap should be in the palm of the hand.

Step 2: Unroll the band closest to the fingers and secure it with the velcro tab to the hand wrap with firm and comfortable compression. The top edge of the hand wrap should line up with the base of the fingers.

Step 3: Wrap the remaining band around the wrist and secure the Velcro tab. Be sure not to over tighten the wrist band so that fluid can freely move out of the hand. The circaid juxtafit essentials hand wrap should overlap with the circaid juxtafit essentials arm.

---

**Note:**

Once all bands are secured inspect the garment for any gaps or creases. Adjust the bands as necessary.

---

**Removing the circaid® juxtafit® essentials hand wrap**

Step 1: Detach the bands, starting with the wrist band, and fold them back onto themselves to keep the Velcro tabs lint-free.

Step 2: Detach the inner Velcro tab (palm) and slide the wrap of the hand.

---

**Note**

If using multiple circaid juxtafit garments, only lightly apply bands overlapping the legging below to ensure gradient compression. Set the remaining bands to firm and comfortable compression or prescribed compression level.

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medi UK Ltd.
medi Ukraine LLC