medi USA, LP
6481 Franz Warner Pkwy.
Whitsett, NC 27377 USA
www.mediusa.com

medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth, Germany
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T +32 11 24 25 60
F +32 11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Itapeva 538 cj. 42
Bela Vista, São Paulo - SP
CEP: 01332-000
Brasil
Tel: +55 (11) 3500 8000
sac@medibrasil.com
www.medibrasil.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbæk Strand
Denmark
T +45 70 25 56 10
F +45 70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi.hu
www.medi.hu

medi Italia s.r.l.
Via Giuseppe Ghedini, 2
40069 Zola Predosa (BO)
Italy
T +39 051 613 24 84
F +39 051 613 29 56
info@medi-italia.it
www.medi-italia.it

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Norway AS
P.B. 238
4291 Kopervik
Norway
T +47 52 84 45 00
F +47 52 84 45 09
post@medinorway.no
www.medinorway.no

medi Orient M.E.A. ZCC
TAURUS GROUP
Suite No. M01 Wasel Al Mamzar Bldg.
Al Wuhida Road, Al Mamzar
P.O.BOX NO. 91464
DUBAI, UAE
T: +971 4 2557344
F: +971 4 2557399
md@taurusgroup.net

medi Polska Sp. z o.o.
Zygmunta Starego 26
44-100 Gliwice
Poland
T+48 32 230 60 21
F+48 32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal Lda.
Rua do Centro Cultural, no. 43
1700-106 Lisboa
Portugal
T +351 21 843 71 60
F +351 21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
Business Center NEO GEO
Butlerova st., 17
117342 Moscow
Russia
T +7 (495) 374-04-56
F +7 (495) 374-04-56
info@medirus.ru
www.medirus.ru

medi Sweden AB
Box 6034
192 06 Sollentuna
Sweden
T +46 8 96 97 98
F +46 8 626 68 70
info@medi.se
www.medi.se

4 049772 365678

E012040 / 09.2017

medi


CIRCAID

circaid® juxtafit® essentials
leg garments
inelastic compression system

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circaid® juxtafit® essentials
lower leg

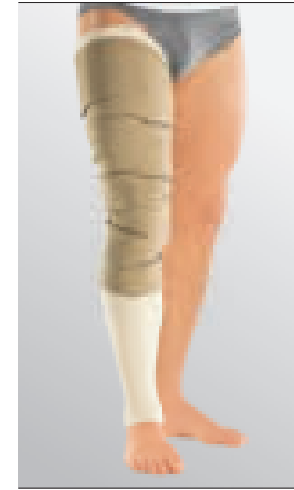circaid® juxtafit® essentials
upper leg with knee

circaid® juxtafit® essentials
upper leg with knee cover

medi. I feel better.

circaid® juxtafit® essentials lower leg

circaid® juxtafit® essentials upper leg

circaid® juxtafit®
essentials leg

Purpose

The garment is single patient, reusable, non-invasive, nonsterile device designed to provide leg compression to patients with lymphatic disorders.

The legging consist of a series of juxtapositioned inelastic bands extending from the central portion of the garment, which contours the leg.

Indications

- Lymphedema
- dependent edema

Relative contraindication

Due to a known contraindication the attending doctor weighs up the wearing of a medical compression garment (for a suitable indication such as chronic venous insufficiency, etc.), and compares the possible risk with the therapeutic benefit of the medical device.

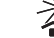
- Decreased or absent sensation in the leg
- Intolerance to compression material

Absolute contraindication

A condition that completely forbids wearing the medical compression garment despite a suitable indication (e.g. chronic venous insufficiency, etc.).

- Severe peripheral arterial disease
- Decompensated congestive heart failure
- Septic phlebitis
- Phlegmasia cerula dolens
- Moderate peripheral arterial disease
- Infection in the leg and / or ankle and / or foot

Storage instructions

-  Store in a dry place
-  Keep out of sunlight

Lifespan

Due to material wear-and-tear and the medical efficacy can only be guaranteed for a defined lifespan.

This assumes correct handling of the garment (e.g. proper care, donning, and doffing).

circaid juxtafit essentials lower leg / circaid juxtafit essentials upper leg with knee / circaid juxtafit essentials knee cover: 6 month

Disposal

 Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

Material composition

circaid juxtafit essentials upper leg with knee:

- 54% Polyurethane
- 42% Nylon
- 4 % Elastane

circaid juxtafit essentials lower leg:

- 53% Polyurethane
- 43% Nylon
- 4% Elastane

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
United Kingdom
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Tankova Str. 8, office 35
Business-center "Flora Park"
Kiev 04112
Ukraine
T +380 44 591 1163
F +380 44 392 73 73
info@medi.ua
http://medi.ua

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000
USA
T +1 336 4 49 44 40
F +1 888 5 70 45 54
info@mediusa.com
www.mediusa.com

medi Trading (Shanghai) Co. Ltd.
Unit 1806, Floor 18, Sandhill Plaza No. 1
2290 Zuchongzhi Road
200131 Shanghai
People's Republic of China
T: +86 1366 1668 687
F: +86 2161 7696 07
infocn@medi.cn

circaid juxtafit essentials knee cover
53% Polyurethane
41% Nylon
4% Elastane
2% Polyethylene

This garment does not contain latex.

Washing instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting.

To extend the life of the garment, hand wash and drip dry.

 Machine wash warm

 Do not bleach

 Tumble dry, low heat

 Do not iron

 Do not dry clean

 Do not wring

Wearing recommendation

Always ensure that the correct prescribed pressure range is being applied.

Slightly loosen the bands of the garment for night-time wear.

Each vertical BPS line on the bands may be at a different location on the leg depending on the size and shape of the leg. They do not need to be aligned down the front center of the leg or in any other specific location on the leg.

If experiencing any pain, immediately remove the garment.

When bands are placed correctly they should slightly overlap, leaving no space between bands.

Do not tuck bands. The Velcro tabs should never touch the skin!

Donning the circaid® juxtafit® essentials lower leg

Step 1: Slide the circaid undersleeve onto the lower leg and over the knee. The cuffed edge of the undersleeve should cover the heel. Ensure that there are no wrinkles in the undersleeve.

Step 2: Align the garment just above the ankle bone and below the knee crease. The black side of the material should face the skin. Unroll the top two bands and loosely secure them to the outside of the garment to anchor the legging in place. This enables you to easily raise or lower the garment for correct positioning.

Note:

The bottom most band can be identified by the circaid-logo on the black side of the material.

Step 3: Unroll the bottom two bands and secure the bottom most band to the outside of the legging to a firm and comfortable compression level.

Step 4: While holding the second band, detach the next band. Secure the second band and continue this process with the other bands, going up the leg.

Step 5: The garment should lay flat and wrinkle free against the leg. Adjust each band to a firm and comfortable compression level. Ensure that the bands are placed in an alternating order.

Step 6: Once all bands are secured, inspect the garment for any gaps or creases. Adjust the bands as necessary.

The legging should never hurt. If any pain is being experienced, the legging

should be removed immediately and a physician should be contacted.

Note:

Once the correct pressure is set using the BPS card, you may fold the top and bottom part of the circaid undersleeve over the top and bottom edge of the circaid juxtafit garment.

Step 7: Donning the circaid compression ankle

First fold the bottom part of the circaid undersleeve over the bottom edge of the circaid garment. Slide the circaid compression ankle onto the foot and ensure that it is pulled high enough so that it overlaps with the circaid juxtafit garment. Ensure that there are no wrinkles in the ankle.

Dofting the circaid® juxtafit® essentials lower leg

To remove the garment, remove the circaid compression ankle first. Then unfold the circaid undersleeve. Detach all bands of the legging starting with the top band. Fold each band onto itself to keep the Velcro tabs lint-free. Remove the garment and then remove the circaid undersleeve.

Donning the circaid® juxtafit® essentials upper leg with knee:

Step 1: Slide the circaid undersleeve whole leg up to the groin. Ensure that there are no wrinkles in the sock.

Step 2: Align the circaid juxtafit upper leg with knee just below the groin. The black side of the material should face the skin. Make sure that the lateral rise of the garment is positioned on the lateral side of the upper leg.

Step 3: Unroll the top two bands and loosely secure them far enough below the groin as to not cause discomfort.

This will anchor the legging in place and enable you to easily raise or lower the garment for correct positioning.

Step 4: Unroll the bottom two bands and secure the bottom most band to the outside of the legging to a firm and comfortable compression level.

Step 5: While holding the second band, detach the next band. Secure the second band and continue this process with the other bands, going up the leg.

Step 6: The garment should lay flat and wrinkle free against the leg. Adjust each band to a firm and comfortable compression level. Ensure that the bands are placed in an alternating order.

Step 7: Once all bands are secured, inspect the garment for any gaps or creases. Adjust the bands as necessary.

The legging should never hurt. If any pain is being experienced, the legging should be removed immediately and a physician should be contacted.

Note:

Once the correct pressure is set using the BPS card, you may fold the top and bottom part of the circaid undersleeve over the top and bottom edge of the circaid juxtafit garment.

Recommendation for combination of lower and upper leg with knee garments:

If combining a circaid juxtafit essentials lower leg with a circaid juxtafit essentials upper leg with knee, ensure that the upper leg with knee overlaps with the lower leg.

Doffing the circaid® juxtafit® essentials upper leg with knee:

Step 1: Detach all bands of the legging starting with the top band. Fold each band onto itself to keep the Velcro tabs lint-free.

Step 2: Remove the garment and then remove the circaid undersleeve.

Donning the circaid® juxtafit® essentials knee cover:

For additional compression in the knee area apply the circaid juxtafit essentials knee cover. Place the circaid juxtafit essentials knee cover across the knee cap and attach it to the garment. Pull the bands in varying directions to achieve a snug fit across the knee.

Doffing the circaid® juxtafit® essentials knee cover:

To remove the circaid juxtafit essentials knee cover detach all Velcro tabs and remove the knee cover from the garment.

Using the Built-In-Pressure card:

It is recommended to always use the BPS card packaged with the product.

Step 1: Locate the Built-In-Pressure card in your packaging.

Step 2: Starting with the bottom band, line up the black triangle on the card with one of the BPS lines on the bottom band.

Step 3: Note where the second BPS line lines up with the card's compression ranges.

Step 4: If the BPS line on the garment either falls short (Fig. B) or goes beyond the correct compression range (Fig. C), readjust the band as necessary so that


the second BPS line on the garment is aligned with the prescribed compression range (Fig. A). Completely loosen the band before reapplying it if too much tension has been applied.

Step 5: Repeat the steps 2 - 4 for each band going up the leg. Be sure to adjust all bands to the same compression level. Bands may need adjusting throughout the day to maintain the prescribed compression range.


Note

If using multiple circaid juxtafit made-to-measure garments, only lightly apply bands overlapping the legging below to ensure gradient compression. Set the remaining bands to firm and comfortable compression or prescribed compression level.


A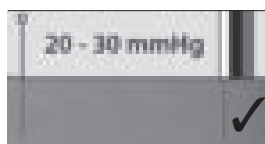

B

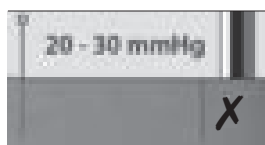C

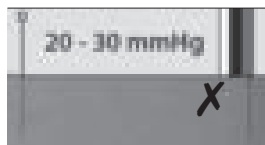Important information for practitioners and patients:

Please note it is recommended that the limb is always treated in its entirety with medical compression.

This means that compression products for the leg begin at the foot or the ankle and extend up to below the hollow of the knee or beyond to include the thigh. For a variety of reasons, the compression garment may be divided into individual segments, e.g. for production reasons, to enhance treatment, compliance, or for easier handling.

A compression garment exclusively for the upper leg or knee for the indications listed is only appropriate when used under the direct supervision and on the recommendation of a Physician. In certain circumstances where improperly prescribed or applied, a partial limb compression can damage the patient's health or lead to constriction or stasis.

Pictures of products that, for instance, show a compression garment on the thigh only are, in this case, solely for the sake of clarity (product details, pattern, attachment, clear illustration of donning and doffing), and are not a guide to medical/therapeutic care.

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