medi USA, LP
6481 Franz Warner Pkwy.
Whitsett, NC 27377 USA
www.mediusa.com

medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth, Germany
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61 2 9890 8696
F +61 2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43-512 57 95 15
F +43-512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T: +32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Itapeva 538 cj. 42
Bela Vista, São Paulo - SP
CEP: 01332-000
Brasil
T +55 (11) 3500 8000
sac@medibrasil.com
www.medibrasil.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbaek Strand
Denmark
T +45 70 25 56 10
F +45 70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi.hu
www.medi.hu

medi Italia s.r.l.
Via Giuseppe Ghedini, 2
40069 Zola Predosa (BO)
Italy
T: +39-051 6 13 24 84
F: +39-051 6 13 29 56
info@medi-italia.it
www.medi-italia.it


medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Norway AS
P.B. 238
4291 Kopervik
Norway
T: +47 52 84 45 00
F: 47 52 84 45 09
post@medinorway.no
www.medinorway.no


4 049772 764662 E012261/09.2017

medi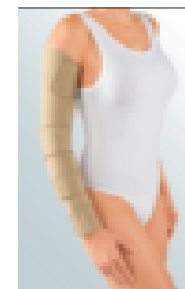

circaid® juxtafit®
essentials arm

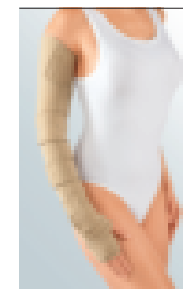circaid® juxtafit®
essentials arm
with
circaid® juxtafit®
essentials hand
wrap

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Step 2

Step 3

circaid® juxtafit® essentials elbow strap

Step 1

circaid® juxtafit® essentials arm

Purpose

This garment is a single patient, reusable, non-invasive, non-sterile device designed to provide compression to the arm of patients with lymphatic disorders. The armsleeve consists of a series of juxtapositioned inelastic bands extending from the central portion of the wrap. It is designed to cover the arm from the wrist to the axilla.

Additionally there is a hand wrap included, which provides compression to the hand, from the wrist to the base of the fingers.

Indications

- Lymphedema

Relative contraindication

Due to a known contraindication the attending doctor weighs up the wearing of a medical compression garment (for a suitable indication such as Lymphedema, etc.), and compares the possible risk with the therapeutic benefit of the medical device.

- Decreased or absent sensation in the arm
- Allergy to compression materials

Absolute contraindication

A condition that completely forbids wearing the medical compression garment despite a suitable indication (e.g. Lymphedema, etc.).

- Severe peripheral arterial disease

- Decompensated congestive heart failure
- Septic phlebitis
- Phlegmasia cerula dolens
- Moderate peripheral arterial disease
- Acute Infection

Storage instructions

Store in a dry place

Keep out of sunlight

Lifespan

Due to material wear-and-tear and the medical efficacy can only be guaranteed for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, donning, and doffing).

circaid juxtafit essentials arm: 6 month.

Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

Material composition

circaid juxtafit essentials arm
63% Nylon
32% Polyurethane
5% Elastane

circaid juxtafit essentials elbow strap
55% Nylon
40% Polyurethane
5% Elastane

medi Orient M.E.A. ZCC
TAURUS GROUP
Suite No. M01 Wasel Al Mamzar Bldg.
Al Wuhida Road, Al Mamzar
P.O.BOX NO. 91464
DUBAI, UAE
United Arab Emirates
T: +971-4 2557344
F: +971- 4 2557399
md@taurusgroup.net

medi Polska Sp. z o.o.
Zygmunta Starego 26
44-100 Gliwice
Poland
T +48-32 230 60 21
F +48-32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal Lda.
Rua do Centro Cultural, no. 43
1700-106 Lisbon
Portugal
T +351 21 843 71 60
F +351 21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
Business Center NEO GEO
Butlerova st., 17
117342 Moscow
Russia
T +7 (495) 374-04-56
F +7 (495) 374-04-56
info@medirus.ru
www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
Unit 1806, Floor 18, Sandhill Plaza No.1
2290 Zuchongzhi Road
200131 Shanghai
People's Republic of China
T: +86-1366 1668 687
F: +86-2161 7696 07
infocn@medi.cn

medi Sweden AB
Box 6034
192 06 Sollentuna
Sweden
T +46 8 96 97 98
F +46 8 626 68 70
info@medi.se
www.medi.se

Medi Turk Ortopedi Medikal
Ic ve Dis Tic. AS
Kültür mah. caldiran sok.14/3
06420 Çankaya/Ankara
Turkey
T +90 312 435 20 26
F +90 312 434 22 82
info@medi-turk.com
www.medi-turk.com

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
Great Britain
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Tankova Str. 8, office 35
Business-center „Flora Park“
Kiev 04112
Ukraine
T +380 44 591 11 63
F +380 44 392 73 73
info@medi.ua
http://medi.ua

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000
USA
T +1-336 4 49 44 40
F +1-888 5 70 45 54
info@mediusa.com
www.mediusa.com

This garment does not contain latex.

Washing instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting.

To extend the life of the garment, hand wash and drip dry.

 Machine wash warm

 Do not bleach

 Tumble dry, low heat

 Do not iron

 Do not dry clean

 Do not wring

Wearing recommendation

Always ensure that the correct prescribed pressure range is being applied. If desired, temporarily detach the elbow band in order to increase mobility. Slightly loosen the bands of the garment for night-time wear. Each vertical BPS line on the bands may be at a different location on the arm depending on the size and shape of the arm. They do not need to be aligned down the front center of the arm or in any other specific location on the arm. If experiencing any pain, immediately remove the garment.

When bands are placed correctly they should slightly overlap, leaving no space between bands. Do not tuck bands. The Velcro tabs should never touch the skin!

Donning circaid® juxtafit® essentials arm:

Step 1: Don optional circaid undersleeve. The circaid juxtafit

essentials arm can also be worn directly against the skin.

Step 2: Slide the circaid juxtafit essentials arm onto the arm. The top of the armsleeve should stop comfortably below the axilla. The elastic material should be on the inner side of the arm with the elbow section of the sleeve centered with the elbow crease.

Step 3: Unroll the band closest to the wrist, pull it across the elastic section of the sleeve and attach it onto the opposite side with firm and comfortable compression. Repeat this process with every band continuing up the arm. The garment should lay flat and wrinkle-free against the arm. Adjust each band to a firm and comfortable compression level. Ensure that the bands are placed in an alternating order.

Note:

Once all bands are secured inspect the garment for any gaps or creases. Adjust the bands as necessary.

Note

The circaid juxtafit essentials elbow strap will be pre-applied in its proper position for ease of use however some adjusting to maximize comfort and coverage may be needed when wearing the arm garment.

Step 1: Locate the elbow crease.

Step 2: Attach the two Velcro tabs of one side of the elbow strap to the arm garment next to the elbow crease.

Step 3: Pull the elbow strap across the elbow crease and secure the remaining two Velcro

tabs to the other side of the garment.

Ensure that the Velcro tabs never make contact with the thin, elastic material.

If desired, temporarily remove the elbow strap to increase mobility. The circaid juxtafit essentials elbow strap is symmetrical, so either end may be used for alignment.

Doffing circaid® juxtafit® essentials arm

Step 1: To remove the circaid juxtafit essentials arm, detach all bands starting with the top band. Fold each band onto itself to keep the Velcro tabs lint-free. Remove the garment.

Step 2: Remove the garment.

Note

If you have decided to wear a circaid undersleeve beneath the garment, remove the undersleeve.

Using the Built-In-Pressure card:

It is recommended to always use the BPS card packaged with the product.

Step 1: Locate the Built-In-Pressure card in your packaging.

Step 2: Starting with the bottom band, line up the black triangle on the card with one of the BPS lines on the bottom band.

Step 3: Note where the second BPS line lines up with the card's compression ranges.

Step 4: If the BPS line on the garment either falls short (Fig. B)

or goes beyond the correct compression range (Fig. C), readjust the band as necessary so that the second BPS line on the garment is aligned with the prescribed compression range (Fig. A). Completely loosen the band before reapplying it if too much tension has been applied.

Step 5: Repeat the steps 2 - 4 for each band going up the leg. Be sure to adjust all bands to the same compression level. Bands may need adjusting throughout the day to maintain the prescribed compression range.


Note

If using multiple circaid juxtafit made-to-measure garments, only lightly apply bands overlapping the legging below to ensure gradient compression. Set the remaining bands to firm and comfortable compression or prescribed compression level.


A

B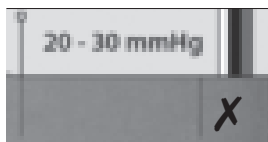

C

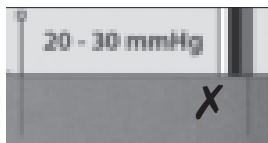Important information for practitioners and patients:

Please note that the limb is always treated in its entirety with medical compression.

This means that compression products for the arm begin at the hand or the wrist, and some extend up to just below the elbow or beyond to below the axilla.

For a variety of reasons, the compression garment may be divided into individual segments, e.g. for production reasons, to enhance treatment compliance, or for easier handling.

A compression garment exclusively for the upper arm for the indications only appropriate when used under the direct supervision and on the recommendation of a Physician. In certain circumstances where improperly prescribed or applied, a partial limb compression can damage the patient's health or lead to constriction or stasis.

Pictures of products that, for instance, show a compression garment on the upper arm only are, in this case, solely for the sake of clarity (product details, pattern, attachment, clear illustration of donning and doffing), and are not a guide to medical/therapeutic care.

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circaid juxtafit essentials Arm:
6 Monate

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circaid juxtafit essentials Arm
63% Nylon
32% Polyurethan
5% Elasthan

circaid juxtafit essentials
Armbeugenband
55% Nylon
40% Polyurethan
5% Elasthan

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