medi USA, LP
6481 Franz Warner Pkwy.
Whitsett, NC 27377 USA
www.mediusa.com

medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth, Germany
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Hungary Kft.
Bokor u. 21
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi.hu
www.medi.hu

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T +32 11 24 25 60
F +32 11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Italia s.r.l.
Via Giuseppe Ghedini, 2
40069 Zola Predosa (BO)
Italy
T +39 051 613 24 84
F +39 051 613 29 56
info@medi-italia.it
www.medi-italia.it

medi Brasil
Rua Itapeva 538 cj. 42
Bela Vista, São Paulo - SP
CEP: 01332-000
Brasil
Tel: +55 (11) 3500 8000
sac@medibrasil.com
www.medibrasil.com

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbæk Strand
Denmark
T +45 70 25 56 10
F +45 70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Norway AS
P.B. 238
4291 Kopervik
Norway
T +47 52 84 45 00
F +47 52 84 45 09
post@medinorway.no
www.medinorway.no

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

4 049772 365593

EO12032/11.2017

medi

circaid juxtafit premium
interlocking ankle foot wrap


circaid juxtafit premium
ankle foot wrap

CIRCAID

circaid® juxtafit® premium
foot options
inelastic compression system

Instructions for use. Gebrauchsanweisung.
Инструкция по использованию. Instruções de uso.
Instrucciones de uso. Mode d'emploi. Gebruiksaanwijzing.
Istruzioni per l'uso. Bruksanvisning. Brugsvejledning.
Instrukcja użytkowania

medi. I feel better.

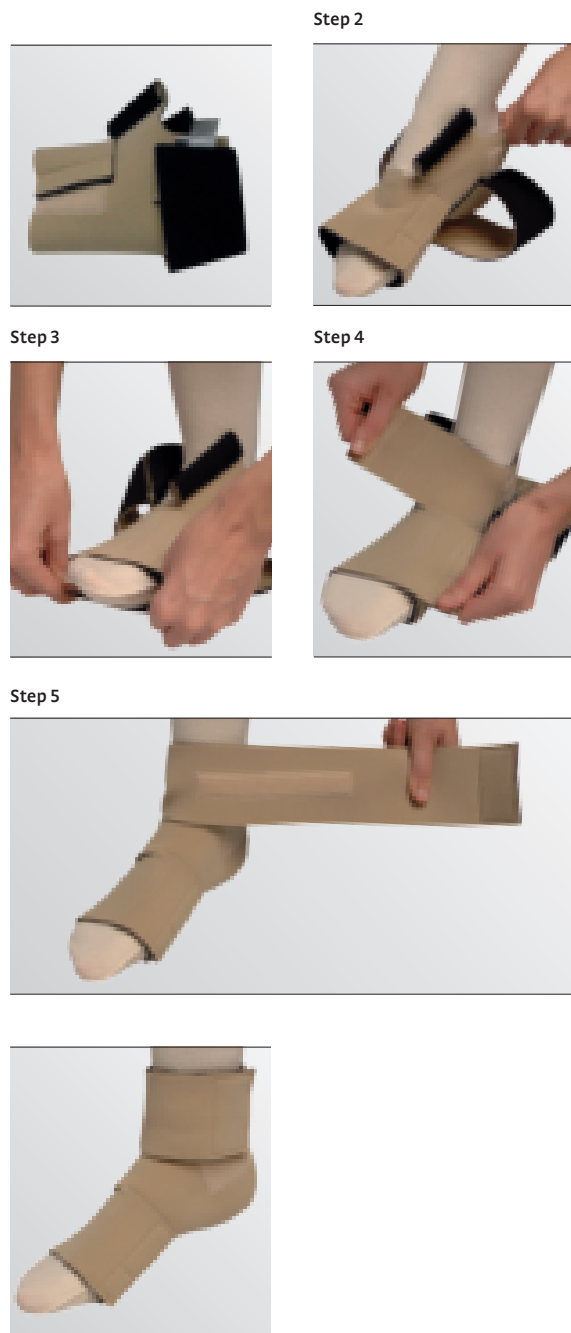Step 2

Step 4

Step 3

Step 5

circaid® juxtafit® premium foot options

Purpose

The foot wrap is a single patient, reusable, non-invasive, non-sterile device designed to provide foot and ankle compression to patients with venous and lymphatic disorders. The foot wrap consist of a series of juxtaposed, inelastic bands extending from a central portion of the wrap. The wrap contours the foot and is designed to cover from the base of the toes to just above the ankle, overlapping with the legging.

Indications

- Lymphedema
- Other forms of edema
- Lipedema
- Varicose veins
- Chronic Venous Insufficiency (CVI)
- Venous stasis disease
- Venous Valvular Insufficiency (VVI)
- Venous insufficiency
- Post-thrombotic syndrome
- Venous stasis ulcer
- Angiodysplasia
- Venous eczema
- Lipodermatosclerosis

Relative contraindication

Due to a known contraindication the attending doctor weighs up the wearing of a medical compression garment (for a suitable indication such as chronic venous insufficiency, etc.), and compares the possible risk with the therapeutic benefit of the medical device.

- Decreased or absent sensation in the leg
- Intolerance to compression material

Absolute contraindication

A condition that completely forbids wearing the medical compression garment despite a suitable indication (e.g. chronic venous insufficiency, etc.).

- Moderate and/or severe peripheral arterial disease
- Septic phlebitis
- Phlegmasia cerula dolens
- Decompensated congestive heart failure
- Infection in the ankle and / or foot

Storage instructions

Store in a dry place

Keep out of sunlight

Lifespan

Due to material wear-and-tear and the medical efficacy can only be guaranteed for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, donning, and doffing).

circaid juxtafit premium ankle foot wrap / circaid juxtafit premium interlocking ankle foot wrap:
<3 months - depends on the wear and tear

Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

medi Orient M.E.A. ZCC
TAURUS GROUP
Suite No. M01 Wasel Al Mamzar Bldg.
Al Wuhida Road, Al Mamzar
P.O. BOX NO. 91464
DUBAI, UAE
T: +971 4 2557344
F: +971 4 2557399
md@taurusgroup.net

medi Polska Sp. z o. o.
Zygmunta Starego 26
44-100 Gliwice
Poland
T+48 32 230 60 21
F+48 32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal Lda.
Rua do Centro Cultural, no. 43
1700-106 Lisboa
Portugal
T +351 21 843 71 60
F +351 21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
Business Center NEO GEO
Butlerova st., 17
117342 Moscow
Russia
T +7 (495) 374-04-56
F +7 (495) 374-04-56
info@medirus.ru
www.medirus.ru

medi Sweden AB
Box 6034
192 06 Sollentuna
Sweden
T +46 8 96 97 98
F +46 8 626 68 70
info@medi.se
www.medi.se

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
United Kingdom
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Tankova Str. 8, office 35
Business-center "Flora Park"
Kiev 04112
Ukraine
T +380 44 591 1163
F +380 44 392 73 73
info@medi.ua
http://medi.ua

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000
USA
T +1 336 4 49 44 40
F +1 888 5 70 45 54
info@mediusa.com
www.mediusa.com

medi Trading (Shanghai) Co. Ltd.
Unit 1806, Floor 18, Sandhill Plaza No.1
2290 Zuchongzhi Road
200131 Shanghai
People's Republic of China
T: +86 1366 1668 687
F: +86 2161 7696 07
info@medi.cn

Material composition

circaid juxtafit premium
interlocking ankle foot wrap/
circaid juxtafit premium ankle
foot wrap:
Polyurethane
Nylon
Elastane
Polyethylene

This garment does not contain latex.

Washing instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. If doing so, it is recommended that you place the garment in a pillow case to aid in keeping the Velcro tab material lint-free and to protect other clothing from getting damaged. To extend the life of the garment, hand wash and drip dry.

 Machine wash warm

 Do not bleach

 Tumble dry, low heat

 Do not iron

 Do not dry clean

 Do not wring

Wearing recommendation

Slightly loosen the bands of the garment for night-time wear. If experiencing discomfort while lying down it is recommended to remove the ankle foot wrap.

If experiencing any pain, immediately remove the foot wrap.

When bands are placed correctly they should slightly overlap, leaving no space between bands.

Do not tuck bands. The Velcro tabs should never touch the skin!

Customising instructions

This product is not designed to be cut or modified except as outlined in the directions for use.

Be careful that the reinforced seam within the hook material is not cut while shortening the juxtafit premium afw.

1* heel cup

2* Notch

3* cut line

4* reinforced cross seam

5* long ankle-band

See details on the foot-length adjustment.

The circaid juxtafit ankle foot wrap can be tailored to fit the patient's foot size and contour. The ankle foot wrap should be fitted by a medical professional. Doing other than instructed will void the three month limited warranty.

Donning the circaid® juxtafit® premium interlocking ankle foot wrap:

Step 1:

Apply a circaid sock first. Ensure that there are no wrinkles in the sock.

Step 2:

Slide the foot into the circaid juxtafit interlocking ankle foot wrap. Unroll the two bands closest to the toes and secure them to the opposing side with firm and comfortable compression. While attaching the second band, pull the ankle-band into the opposing direction.

Step 3:

Ensure that the foot is properly aligned within the foot wrap. The

front edge of the foot wrap should align with the base of the toes. Adjust the interlocking heel-bands to fit snugly around the back of the heel and attach them to the landing pads with firm and comfortable compression.

Step 4:
Detach the ankle-band and pull it around the ankle with firm and comfortable compression. Attach it to the landing pad. Ensure that there are no gaps. Adjust the ankle-band if any gaps or pressure points occur. The ankle-band should overlap with the circaid leg garment.

Note:

The ankle-band may pull from a different side depending on which foot, left or right, the garment is applied to.

Step 5:
Readjust the bands as necessary throughout the day to maintain a firm and comfortable compression level.

Removal of the circaid® juxtafit® premium interlocking ankle foot wrap:

First detach the ankle band and fold it back onto itself to keep it lint free. Lastly, loosen the two bands closest to the toes and roll them back onto themselves as well. Do not remove the heel bands for easy reapplication.

Donning the circaid® juxtafit® premium ankle foot wrap:

Step 1:

Apply a circaid sock first. Ensure that there are no wrinkles in the sock.

Step 2:

Unroll the band closest to the toes and secure it to the opposing side to create a kind of slipper. Slide the foot into the slipper shape and firmly press the heel into the heel cup.

Step 3:

The front edge of the foot options should align with the base of the toes.

Step 4:

Tighten the band closest to the toes to firm and comfortable compression.

Step 5:

While pulling the ankle band in the opposing direction, attach the second band on the opposing landing pad with firm and comfortable compression. If necessary, adjust the band closest to the toes to firm and comfortable compression. Adjust the ankle band if any gaps or pressure points occur. The ankle-band should overlap with the circaid leg garment.

Note:

The ankle band may pull from a different side depending on which foot, left or right, the garment is applied to.

Step 6: Readjust the bands as necessary throughout the day to maintain a firm and comfortable compression level.

The garment should never hurt. It should feel firm but comfortable.

If pressure increases or decreases noticeably during wear, readjust the garment as needed. If experiencing any pain,

immediately remove the garment and consult your physician.

**Removal of the circaid®
juxtafit® premium ankle foot
wrap:**

First detach the ankle band and the second band and fold it back onto itself to keep it lint free. Lastly, loosen the band closest to the toes. The circaid juxtafit premium ankle foot wrap slips off like a shoe.

**Foot-length adjustment of the
circaid® juxtafit® premium
ankle foot wrap:**

The following steps will permanently reduce the foot-length of the circaid juxtafit premium ankle foot wrap. The afw can be trimmed by up to 1.25" (3 cm) (see notches on both sides).

Step 1:

With the beige side facing up, mark the desired length.

Step 2:

Draw a straight line connecting the notches.

Step 3: Carefully cut along the drawn line, assuring that the reinforced seam section of the Velcro material is not cut.

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