medi USA, LP
6481 Franz Warner Pkwy.
Whitsett, NC 27377
USA
www.mediusa.com

medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth, Germany
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61 2 9890 8696
F +61 2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbaek Strand
Denmark
T +45 70 25 56 10
F +45 70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43-512 57 95 15
F +43-512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T: +32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medi belgium.be

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Brasil
Rua Cristovan de Vita, n.º 260, Un. 30,
Centro Logístico Raposo Tavares,
Vila Camargo
Vargem Grande Paulista - SP
CEP 06730-000
Brasil
T: +55-11-3500 8005
sac@medibrasil.com
www.medi brasil.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi.hu
www.medi.hu

medi Canada Inc / médi Canada Inc
597, Rue Duvernay, Verchères
QC Canada J0L 2R0
T: +1 450-583-3317 / +1 800 361 3153
F: +1 888-583-6827
service@medicanada.ca
www.medicana da.ca

medi Italia s.r.l.
Via Giuseppe Ghedini, 2
40069 Zola Predosa (BO)
Italy
T: +39-051 6 13 24 84
F: +39-051 6 13 29 56
info@medi-italia.it
www.medi-italia.it


MAXIS a.s.,
medi group company
Slezska 2127/13
120 00 Prague 2
Czech Republic
T: +420 571 633 510
F: +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

E012039/08.2019

medi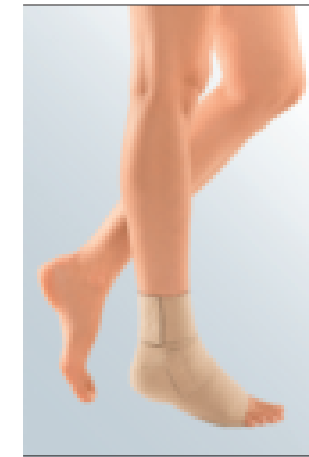

circaid juxtalite ankle foot wrap

CIRCAID

circaid® juxtalite®
ankle foot wrap
inelastic compression system

Instructions for use. Gebrauchsanweisung.
Инструкция по использованию. Instruções de uso.
Instrucciones de uso. Mode d'emploi. Gebruiksaanwijzing.
Istruzioni per l'uso. دليل الاستخدام. Bruksanvisning.
Brugsvejledning. הוראות שימוש.

medi. I feel better.

Step 1

Step 2

Step 3


Step 4

Step 5

Step 6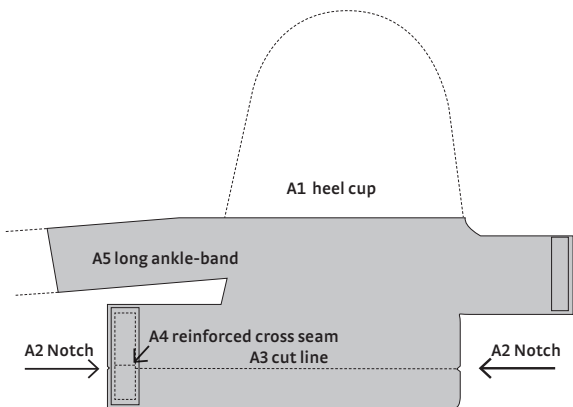

circaid® juxtalite® ankle foot wrap

Purpose

The foot wrap is a single patient, reusable, non-invasive, non-sterile device designed to provide foot and ankle compression to patients with venous and lymphatic disorders. The foot wrap consist of a series of juxtaped, inelastic bands extending from a central portion of the wrap. The wrap contours the foot and is designed to cover from the base of the toes to just above the ankle, overlapping with the legging.

Indications

- CVI
- Varicose veins
- DVT/Thrombosis Prevention
- Post Thrombotic Syndrome
- Venous Stasis Ulcer
- Post Sclerotherapy
- Dependent oedema

Contraindications

- Severe Peripheral Arterial Disease
- Uncontrolled Congestive Heart Failure
- Septic Phlebitis
- Phlegmasia Cerulea Dolens
- Untreated infection
- Any circumstance where increased venous and lymphatic return is undesirable
- Suspected or known untreated acute DVT

Cautions

- Mild to Moderate Peripheral Arterial

Disease

- Impaired Sensation - must be able to sense applied pressure
- Intolerance to material(s)
- Pediatric & assisted use - must be able to sense & communicate applied pressure

Storage instructions

-  Store in a dry place
-  Keep out of sunlight

Customising instructions

circaid juxtalite ankle foot wrap can be tailored to fit the patient's foot length. The ankle foot wrap should be fitted by a medical professional. Doing other than instructed will void the three month limited warranty.

See more details in the section ankle-band and foot-length adjustment.

Lifespan:

Due to material wear-and-tear and the medical efficacy can only be guaranteed for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, donning, and doffing).

circaid juxtalite ankle foot wrap:
<3 month - depends on the wear and tear

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Norway AS
Vestheimvegen 35
4250 Kopervik
Norway
T: +47 52 84 45 00
F: 47 52 84 45 09
post@medinorway.no
www.medinorway.no

medi Polska Sp. z o.o.
ul. Łąbiedzka 22
44-121 Gliwice
Poland
T: +48-32 230 60 21
F: +48-32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal Lda.
Rua do Centro Cultural, no. 43
1700-106 Lisbon
Portugal
T +351 21 843 71 60
F +351 21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
Business Center NEO GEO
Butlerova st., 17
117342 Moscow
Russia
T +7 (495) 374-04-56
F +7 (495) 374-04-56
info@medirus.ru
www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
Room 105, Building No. 4,
Lane 500 ShengXia Road
Pudong New District
201210 Shanghai
People's Republic of China
T: +86-21 61761988
F: +86-21 61769607
info@medi.cn

medi Sweden AB
Box 6034
192 06 Sollentuna
Sweden
T +46 8 96 97 98
F +46 8 626 68 70
info@medi.se
www.medi.se

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
Great Britain
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Evhena Sverstiuka str, 11.
Kiev 02002
Ukraine
T: +380 44 591 11 63
F: +380 44 392 73 73
info@medi.ua
http://medi.ua

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000
USA
T +1-336 4 49 44 40
F +1-888 5 70 45 54
info@mediusa.com
www.mediusa.com

Medi Turk Ortopedi Medikal Iç ve Dis Tic. AS
Kültür mah. çaldıran sok.14/3
06420 Çankaya/Ankara
Turkey
T: +90 312 435 20 26
F: +90 312 434 22 82
info@medi-turk.com
www.medi-turk.com

Tachezy Sanit Srl.
Via Druso 48/a
39100 Bolzano
Italy
T: +39 0471 288 068
F: +39 0471 282 773
info@tachezysanit.com
www.tachezysanit.com

Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

Material composition

circaid juxtalite ankle foot wrap
56% Nylon
39% Polyurethane
5% Elastane

This garment does not contain latex.

Washing instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. If doing so, it is recommended that you place the garment in a pillow case to aid in keeping the Velcro tab material lint-free and to protect other clothing from getting damaged. To extend the life of the garment, hand wash and drip dry.

 Machine wash warm

 Do not bleach

 Tumble dry, low heat

 Do not iron

 Do not dry clean

 Do not wring

Wearing recommendation

Slightly loosen the bands of the garment for night-time wear. If experiencing discomfort while lying down it is recommended to remove the ankle foot wrap.

If experiencing any pain, immediately remove the foot wrap.

When bands are placed correctly they should slightly overlap, leaving no space between bands.

Do not tuck bands. The Velcro tabs should never touch the skin!

Donning

Step 1: Don optional circaid sock or undersleeve. The circaid juxtalite ankle foot wrap (afw) can also be worn directly against the skin.

Step 2: First, unroll the band closest to the toes and secure it to the opposing side to create a kind of slipper. Slide the foot into the slipper shape and firmly press the heel into the heel cup.

Step 3:
The front edge of the foot piece should align with the base of the toes.

Step 4:
Tighten the band closest to the toes to firm and comfortable compression. While pulling the ankle band in the opposing direction, attach the second band with firm and comfortable compression.

Step 5:
Unroll the ankle-band and pull it around the ankle with firm and comfortable compression. Ensure there are no gaps and fasten the Velcro tab. Adjust the ankle-band if any gaps or pressure points occur.

Note: The ankle-band may pull from a different side depending on which foot, left or right, the afw is applied to. Make sure to secure the ankle-band last. Ensure that the ankle-band overlaps with the circaid leg garment.

Step 6: Readjust the bands as necessary throughout the day to maintain a firm and comfortable compression level.

The garment should never hurt. It should feel firm but comfortable.

If pressure increases or decreases noticeably during wear, readjust the garment as needed. If experiencing any pain, immediately remove the garment and consult your physician.

Removal: First detach the ankle band and fold it back onto itself to keep it lint free. Lastly, loosen the front foot band. The circaid juxtalite ankle foot wrap slips off like a shoe.

Ankle-band adjustment

The following steps will permanently reduce the length of the long ankle-band.

Step 1: Determine and mark the desired length of the ankle-band.

Step 2: Remove the afw and carefully cut it to the desired length.

Step 3: Use the included Velcro tabs to secure the ankle-band in place.

Foot-length adjustment

The following steps will permanently reduce the foot-length of the circaid juxtalite ankle foot wrap. The afw can be trimmed by up to 1.25 in. (3 cm) (see notches on both sides).

Step 1: With the beige side facing up, mark the desired length.

Step 2: Draw a straight line from where you would like to trim.

Step 3: Carefully cut along the drawn line, assuring that the reinforced seam section of the Velcro material is not cut.

Please note: No other modifications can be made to the garment. Be careful that the reinforced seam within the Velcro material is not cut while shortening the garment

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