medi USA, LP
6481 Franz Warner Pkwy.
Whitsett, NC 27377
USA
www.mediusa.com

medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth, Germany
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61 2 9890 8696
F +61 2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43-512 57 95 15
F +43-512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi.hu
www.medi.hu

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T: +32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Italia s.r.l.
Via Giuseppe Ghedini, 2
40069 Zola Predosa (BO)
Italy
T: +39-051 6 13 24 84
F: +39-051 6 13 29 56
info@medi-italia.it
www.medi-italia.it

medi Brasil
Rua Itapeva 538 cj. 42
Bela Vista, São Paulo - SP
CEP: 01332-000
Brasil
T +55 (11) 3500 8000
sac@medibrasil.com
www.medibrasil.com

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbaek Strand
Denmark
T +45 70 25 56 10
F +45 70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Norway AS
P.B. 238
4291 Kopervik
Norway
T: +47 52 84 45 00
F: 47 52 84 45 09
post@medinorway.no
www.medinorway.no

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

4 049772 365609 E012033/09.2017

medi


CIRCAID

circaid® juxtalite®
lower leg
inelastic compression system

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circaid juxtalite lower leg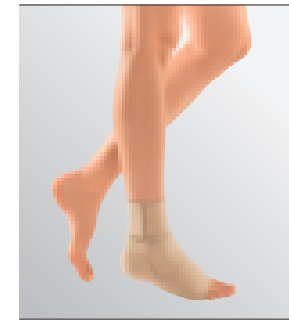

circaid juxtalite ankle foot
wrap

circaid juxtalite with ankle
foot wrap


circaid juxtalite lower leg with
circaid undersleeve lower leg


circaid juxtalite with
undersleeve anklet

medi. I feel better.

Step 1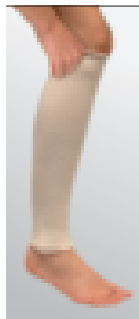


Step 2

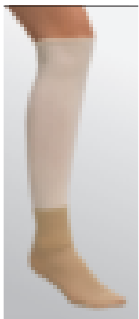Step 3

Step 4

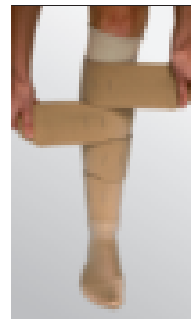Using the Built-In-Pressure card:

Step 5

Step 6

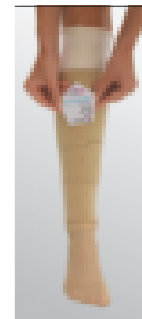
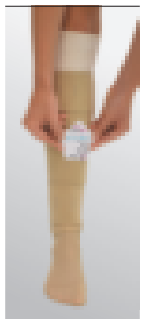
Fig. A

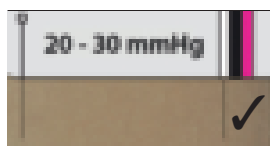
Fig. B


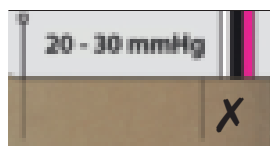
Fig. C

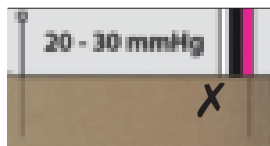circaid® juxtalite® lower leg

Purpose

The garment is a device designed to provide lower leg compression to patients with venous disorders. The legging consists of a series of juxtapositioned, inelastic bands extending from the central portion of the wrap. The wrap contours the leg and is designed to cover from just above the ankle bone to just below the knee.

Indications

- Chronic venous insufficiency
- Venous stasis ulcers
- Post sclerotherapy
- Post thrombotic syndrome
- Dependent edema
- Varicose veins

Relative contraindication

Due to a known contraindication the attending doctor weighs up the wearing of a medical compression garment (for a suitable indication such as chronic venous insufficiency, etc.), and compares the possible risk with the therapeutic benefit of the medical device.

- Decreased or absent sensation in the leg
- Intolerance to compression material

Absolute contraindication

A condition that completely forbids wearing the medical compression garment despite a suitable indication (e.g. chronic venous insufficiency, etc.).

- Severe peripheral arterial disease
- Decompensated congestive heart failure
- Septic phlebitis
- Phlegmasia cerula dolens
- Moderate peripheral arterial disease
- Infection in the ankle and / or the foot

Storage instructions

-  Store in a dry place
-  Keep out of sunlight

The shelf-life date is printed on the carton label with the symbol of a „sand glass“.

Lifespan

Due to material wear-and-tear and the medical efficacy can only be guaranteed for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, donning, and doffing).

circaid juxtalite lower leg:
6 month

Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

medi Orient M.E.A. ZCC
TAURUS GROUP
Suite No. M01 Wasel Al Mamzar Bldg.
Al Wuhida Road, Al Mamzar
P.O.BOX NO. 91464
DUBAI, UAE
United Arab Emirates
T: +971-4 2557344
F: +971- 4 2557399
md@taurusgroup.net

medi Polska Sp. z o.o.
Zygmunta Starego 26
44-100 Gliwice
Poland
T +48-32 230 60 21
F +48-32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal Lda.
Rua do Centro Cultural, no. 43
1700-106 Lisbon
Portugal
T +351 21 843 71 60
F +351 21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
Business Center NEO GEO
Butlerova st., 17
117342 Moscow
Russia
T +7 (495) 374-04-56
F +7 (495) 374-04-56
info@medirus.ru
www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
Unit 1806, Floor 18, Sandhill Plaza No. 1
2290 Zuchongzhi Road
200131 Shanghai
People's Republic of China
T: +86-1366 1668 687
F: +86-2161 7696 07
infocn@medi.cn

medi Sweden AB
Box 6034
192 06 Sollentuna
Sweden
T +46 8 96 97 98
F +46 8 626 68 70
info@medi.se
www.medi.se

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
Great Britain
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Tankova Str. 8, office 35
Business-center „Flora Park“
Kiev 04112
Ukraine
T +380 44 591 11 63
F +380 44 392 73 73
info@medi.ua
http://medi.ua

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000
USA
T +1-336 4 49 44 40
F +1-888 5 70 45 54
info@mediusa.com
www.mediusa.com

Material composition

circaid juxtalite lower leg:

56% Nylon

40% Polyurethane

4% Elastane

This garment does not contain latex.

Washing instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. If doing so, it is recommended that you place the garment in a pillow case to aid in keeping the Velcro tab material lint-free and to protect other clothing from getting damaged.

To extend the life of the garment, hand wash and drip dry.

 Machine wash warm

 Do not bleach

 Tumble dry, low heat

 Do not iron

 Do not dry clean

 Do not wring

Wearing recommendation

Always ensure that the correct prescribed pressure range is being applied.

Slightly loosen the bands of the garment for night-time wear.

Each vertical BPS line on the bands may be at a different location on the leg depending on the size and shape of the leg. They do not need to be aligned down the front center of the leg or in any other specific location on the leg.

If experiencing any pain, immediately remove the garment.

When bands are placed correctly they should slightly overlap, leaving no space between bands.

Do not tuck bands. The Velcro tabs should never touch the skin!

Donning

Step 1: Slide the circaid undersleeve onto the lower leg and over the knee. The cuffed edge of the undersleeve should cover the heel. Ensure that there are no wrinkles in the undersleeve.

Step 2: Align the garment just above the ankle bone and below the knee crease. The black side of the material should face the skin. Unroll the top two bands and loosely secure them to the outside of the garment to anchor the legging in place. This enables you to easily raise or lower the garment for correct positioning.

Note:

The bottom most band can be identified by the circaid-logo on the black side of the material.

Step 3: Unroll the bottom two bands and secure the bottom most band to the outside of the legging to a firm and comfortable compression level.

Step 4: While holding the second band, detach the next band. Secure the second band and continue this process with the other bands, going up the leg.

Step 5: The garment should lay flat and wrinkle free against the leg. Adjust each band to a firm and comfortable compression

level. Ensure that the bands are placed in an alternating order.

Step 6: Once all bands are secured, inspect the garment for any gaps or creases. Adjust the bands as necessary.

The legging should never hurt. If any pain is being experienced, the legging should be removed immediately and a physician should be contacted.

Note:

Once the correct pressure is set using the BPS card, you may fold the top and bottom part of the circaid undersleeve over the top and bottom edge of the circaid juxtalite garment.

Step 7: Donning the circaid compression anklet. First fold the bottom part of the circaid undersleeve over the bottom edge of the circaid garment. Slide the circaid compression anklet onto the foot and ensure that it is pulled high enough so that it overlaps with the circaid juxtalite garment. Ensure that there are no wrinkles in the anklet.

Doffing

To remove the garment, remove the circaid compression anklet first. Then unfold the circaid undersleeve. Detach all bands of the legging starting with the top band. Fold each band onto itself to keep the Velcro tabs lint-free. Remove the garment and then remove the circaid undersleeve.

Using the Built-In-Pressure card:

It is recommended to always use the BPS card packaged with the product.

Step 1: Locate the Built-In-Pressure card in your packaging.

Step 2: Identify the appropriate edge of card and pressure scale. The edge of card is determined by legging size.

Step 3: Starting with the bottom band, line up the vertical line on the card with one of the BPS lines on the bottom band.

Step 4: Note where the second BPS line lines up with the card's compression ranges. (Fig. A, B, and C)

Step 5: If the BPS line on the garment either falls short (Fig. B) or goes beyond the correct compression range (Fig. C), readjust the band as necessary so that the second BPS line on the garment is aligned with the prescribed compression range (Fig. A). Completely loosen the band before reapplying it if too much tension has been applied.

Step 6: Repeat the steps 3 - 5 for each band going up the leg. Be sure to adjust all bands to the same compression level. Bands may need adjusting throughout the day to maintain the prescribed compression range.

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