medi USA, LP  
6481 Franz Warner Pkwy.  
Whitsett, NC 27377  
USA  
www.mediusa.com

EC REP medi GmbH & Co. KG  
Medicusstraße 1  
95448 Bayreuth, Germany  
www.medi.de

medi Australia Pty Ltd  
83 Fennell Street  
North Parramatta NSW 2151  
Australia  
T +61 2 9890 8696  
F +61 2 9890 8439  
sales@mediaustralia.com.au  
www.mediaustralia.com.au

medi Danmark ApS  
Vejlegardsvej 59  
2665 Vallensbaek Strand  
Denmark  
T +45 70 25 56 10  
F +45 70 25 56 20  
kundeservice@sw.dk  
www.medit danmark.dk

medi Austria GmbH  
Adamgasse 16/7  
6020 Innsbruck  
Austria  
T +43-512 57 95 15  
F +43-512 57 95 15 45  
vertrieb@medi-austria.at  
www.medi-austria.at

medi Bayreuth España SL  
C/Canigó 2 – 6 bajos  
Hospitalet de Llobregat  
08901 Barcelona  
Spain  
T +34 93 260 04 00  
F +34 93 260 23 14  
medi@mediespana.com  
www.mediespana.com

medi Belgium bvba  
Posthoornstraat 13/1  
3582 Koersel  
Belgium  
T: +32-11 24 25 60  
F: +32-11 24 25 64  
info@medibelgium.be  
www.medibelgium.be

medi France  
Z.I. Charles de Gaulle  
25, rue Henri Farman  
93297 Tremblay en France Cedex  
France  
T +33 1 48 61 76 10  
F +33 1 49 63 33 05  
infos@medi-france.com  
www.medi-france.com

medi Brasil  
Rua Itapeva 538 cj. 42  
Bela Vista, São Paulo - SP  
CEP: 01332-000  
Brasil  
T +55 (11) 3500 8000  
sac@medibrasil.com  
www.medibrasil.com

medi Hungary Kft.  
Bokor u. 21.  
1037 Budapest  
Hungary  
T +36 1371 0090  
F +36 1371 0091  
info@medi.hu  
www.medi.hu

4 049772 365654

E012038 / 09.2017

medi

CIRCAID

circaid® power added compression™  
(pac band™)  
inelastic compression system

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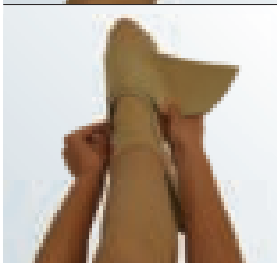
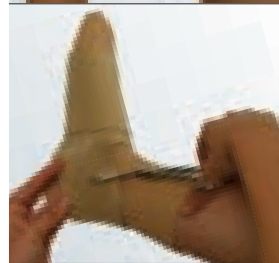
3.1

3.2

## circaid® power added compression™ (pac band™)

### Purpose

The garment is a single patient, reusable, non-invasive, non-sterile device designed to provide compression to the foot and ankle region for patients with venous and lymphatic disorders. It is intended to be used in conjunction with the circaid compression ankle, however can also be used as a standalone product as well.

### Indications

- Lymphedema
- Other forms of edema
  - Venous
  - Post traumatic
  - Post surgical
  - Lipedema
- Varicose veins (all types)
- Chronic venous insufficiency

### (CVI)

- Venous stasis disease
- Venous valvular insufficiency (VVI)
- Venous insufficiency
- Post thrombotic syndrome
- Venous ulcer (stasis ulcer)
- Angiodysplasia
- Post sclerotherapy
- Thrombosis prevention
- Venous eczema
- Lipodermatosclerosis

### Relative contraindication

Due to a known contraindication the attending doctor weighs up the wearing of a medical compression garment (for a suitable indication such as chronic venous insufficiency, etc.), and compares the possible risk with the therapeutic benefit of the medical device.

- Decreased or absent sensation in the leg
- Intolerance to compression material

### Absolute contraindication

A condition that completely

medi Italia s.r.l.  
Via Giuseppe Ghedini, 2  
40069 Zola Predosa (BO)  
Italy  
T: +39-051 6 13 24 84  
F: +39-051 6 13 29 56  
info@medi-italia.it  
www.medi-italia.it

medi Nederland BV  
Heusing 5  
4817 ZB Breda  
The Netherlands  
T +31 76 57 22 555  
F +31 76 57 22 565  
info@medi.nl  
www.medi.nl

medi Norway AS  
P.B. 238  
4291 Kopervik  
Norway  
T: +47 52 84 45 00  
F: 47 52 84 45 09  
post@medinorway.no  
www.medinorway.no

medi Orient M.E.A. ZCC  
TAURUS GROUP  
Suite No. M01 Wasef Al Mamzar Bldg.  
Al Wuhida Road, Al Mamzar  
P.O.BOX NO. 91464  
DUBAI, UAE  
United Arab Emirates  
T: +971- 4 2557344  
F: +971- 4 2557399  
md@taurusgroup.net

medi Polska Sp. z o.o.  
Zygmunta Starego 26  
44-100 Gliwice  
Poland  
T +48-32 230 60 21  
F +48-32 202 87 56  
info@medi-polska.pl  
www.medi-polska.pl

medi Bayreuth Unipessoal Lda.  
Rua do Centro Cultural, no. 43  
1700-106 Lisbon  
Portugal  
T +351 21 843 71 60  
F +351 21 847 08 33  
medi.portugal@medibayreuth.pt  
www.medi.pt

medi RUS LLC  
Business Center NEO GEO  
Butlerova st., 17  
117342 Moscow  
Russia  
T +7 (495) 374-04-56  
F +7 (495) 374-04-56  
info@medirus.ru  
www.medirus.ru

medi Trading (Shanghai) Co. Ltd.  
Unit 1806, Floor 18, Sandhill Plaza No. 1  
2290 Zuchongzhi Road  
200131 Shanghai  
People's Republic of China  
T: +86-1366 1668 687  
F: +86-2161 7696 07  
infocn@medi.cn

medi Sweden AB  
Box 6034  
192 06 Sollentuna  
Sweden  
T +46 8 96 97 98  
F +46 8 626 68 70  
info@medi.se  
www.medi.se

Medi Türk Ortopedi Medikal İc ve Dis Tic. AS  
Kültür mah. Çaldıran sok.14/3  
06420 Çankaya/Ankara  
Turkey  
T +90 312 435 20 26  
F +90 312 434 22 82  
info@medi-turk.com  
www.medi-turk.com

forbids wearing the medical compression garment despite a suitable indication (e.g. chronic venous insufficiency, etc.).

- Severe peripheral arterial disease
- Decompensated congestive heart failure
- Septic phlebitis
- Phlegmasia cerula dolens
- Moderate peripheral arterial disease
- Infection in the leg.

### Storage instructions

Store in a dry place

Keep out of sunlight

### Lifespan

Due to material wear and tear the medical efficacy can only be guaranteed for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, donning, doffing).

circaid power added compression band: <3 month - depends on the wear and tear

### Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

### Material composition

Nylon 67%

Polyurethane 23%

Elastane 5%

Polyethylene 5%

This garment does not contain latex.

### Washing instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. If doing so, it is recommended that you place the garment in a pillow case

to keep the Velcro tab material lint-free and to protect other clothing from getting damaged. To extend the life of the garment, hand wash and drip dry.

---

 Machine wash warm

 Do not bleach

 Drip dry or line dry

 Do not iron

 Do not dry clean

 Do not wring

---

### **Wearing recommendation:**

Slightly loosen the bands of the garment for night-time wear. If experiencing discomfort while lying down it is recommended to remove the foot piece.

If experiencing any pain, immediately remove the foot piece.

Do not tuck bands. The Velcro tabs should never touch the skin!

### **Donning**

The circaid power added compression band (pac band) is

designed to provide additional compression to the foot, targeting the arch and ankle area regions. For complete compression of the foot, it is recommended to apply the cicaid power added compression band over the circaid compression ankle.

**Step 1:** Lay the pac band, out flat. If using on the right foot, the right foot symbol should be visible. If using it on the left foot, the left foot symbol should be visible.

**Step 2:** Center the arch of the foot on the footprint symbol. Pull the outer strap with firm tension to the inner ankle. Hold the strap in place and wrap the inner strap over the dorsum of the foot. Fasten the strap using the larger Velcro tab.

**Step 3:** Pull the remaining strap around the ankle bone with firm and comfortable compression. Secure the strap to the inner foot area of the wrap using the smaller Velcro tab.

Step 4: If excess material or overlap is a concern, either the arch or ankle band may be trimmed as needed or the width of the arch band may be trimmed as needed to fit shorter feet.

Step 5: Readjust the bands as necessary throughout the day to maintain a firm and comfortable compression level.

The garment should never hurt. It should feel firm but comfortable.

If pressure increases or decreases noticeably during wear, readjust the garment as needed.

If experiencing any pain, immediately remove the garment and consult your physician.

### **Removal**

Disengage the Velcro tabs and fold each band back onto itself to keep the Velcro tabs lint-free. Do not tightly roll the band back onto itself as this will cause the hook material to bend, reducing its adherence strength. When complete, the garment should be ready for easy reapplication.

# circaid® power added compression™ (pac band™)

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circaid power added compression band:

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### Inhalt:

1 Stück circaid power added compression band

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