**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**D-95448 Bayreuth**  
**Germany**  
**T +49 921 912-0**  
**F +49 921 912-780**  
**ortho@medi.de**  
**www.medi.de**

medi Australia Pty Ltd  
 83 Fennell Street  
 North Parramatta NSW 2151  
 Australia  
 T +61 2 9890 8696  
 F +61 2 9890 8439  
 sales@mediaustralia.com.au  
 www.mediaustralia.com.au

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43-512 57 95 15  
 F +43-512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium NV  
 Posthoornstraat 13/1  
 3582 Koersel  
 Belgium  
 T: +32-11 24 25 60  
 F: +32-11 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

medi Brasil  
 Rua Cristovao de Vita, n.º 260, Un. 30,  
 Centro Logístico Raposo Tavares,  
 Vila Camargo  
 Vargem Grande Paulista - SP  
 CEP 06730-000  
 Brasil  
 T: +55-11-3500 8005  
 sac@medibrasil.com  
 www.medibrasil.com

medi Canada Inc / médi Canada Inc  
 597, Rue Duvernay, Verchères  
 QC Canada J0L 2R0  
 T: +1 450-583-3317 / +1 800 361 3153  
 F: +1 888-583-6827  
 service@medicanada.ca  
 www.medicanada.ca

MAXIS a.s.,  
 medi group company  
 Slezská 2127/13  
 120 00 Prague 2  
 Czech Republic  
 T: +420 571 633 510  
 F: +420 571 616 271  
 info@maxis-medica.com  
 www.maxis-medica.com

medi Danmark ApS  
 Vejlegardsvej 59  
 2665 Vallensbaek Strand  
 Denmark  
 T +45 70 25 56 10  
 F +45 70 25 56 20  
 kundeservice@sw.dk  
 www.meditandmark.dk

medi Bayreuth España SL  
 C/Canigó 2 – 6 bajos  
 Hospitalet de Llobregat  
 08901 Barcelona  
 Spain  
 T +34 93 260 04 00  
 F +34 93 260 23 14  
 medi@mediespana.com  
 www.mediespana.com

medi France  
 Z.I. Charles de Gaulle  
 25, rue Henri Farman  
 93297 Tremblay en France Cedex  
 France  
 T +33 1 48 61 76 10  
 F +33 1 49 63 33 05  
 infos@medi-france.com  
 www.medi-france.com

medi Hungary Kft.  
 Bokor u. 21.  
 1037 Budapest  
 Hungary  
 T +36 1371 0090  
 F +36 1371 0091  
 info@medi.hu  
 www.medi.hu

medi Japan K.K.  
 5-1 Nihonbashi Kabutocho Chuoku  
 103-0026 Tokyo  
 Japan  
 T: +81 3 6778 2590  
 F: +81 3 5847 7901  
 info@medi-japan.jp  
 www.medi-japan.co.jp

medi Nederland BV  
 Heusing 5  
 4817 ZB Breda  
 The Netherlands  
 T +31 76 57 22 555  
 F +31 76 57 22 565  
 info@medi.nl  
 www.medi.nl

medi Norway AS  
 Vestheimvegen 35  
 4250 Kopervik  
 Norway  
 T: +47 52 84 45 00  
 F: +47 52 84 45 09  
 post@medinorway.no  
 www.medinorway.no

medi Polska Sp. z o.o.  
 ul. Łabedzka 22  
 44-121 Gliwice  
 Poland  
 T: +48-32 230 60 21  
 F: +48-32 202 87 56  
 info@medi-polska.pl  
 www.medi-polska.pl

medi Bayreuth Unipessoal Lda.  
 Rua do Centro Cultural, no. 43  
 1700-106 Lisbon  
 Portugal  
 T +351 21 843 71 60  
 F +351 21 847 08 33  
 medi.portugal@medibayreuth.pt  
 www.medi.pt

medi RUS LLC  
 Business Center NEO GEO  
 Butlerova Street 17  
 117342 Moscow  
 Russia  
 T +7-495 374 04 56  
 F +7-495 374 04 56  
 info@medirus.ru  
 www.medirus.ru

medi Trading (Shanghai) Co. Ltd.  
 Room 1703, T1, CES WEST BUND CENTER,  
 No. 277, Long Lan Road  
 Xuhui District  
 200232 Shanghai  
 People's Republic of China  
 T: +86-21 50582319  
 F: +86-21 50582319

medi Sweden AB  
 Box 6034  
 192 06 Sollentuna  
 Sweden  
 T +46 8 96 97 98  
 F +46 8 626 68 70  
 info@medi.se  
 www.medi.se

Medi Turk Ortopedi Medikal Ic ve Dis Tic. AS  
 Mustafa Kemal Mah. 2156 Sokak No: 16 /4- 5  
 06510 Çankaya Ankara  
 Turkey  
 T: +90 312 435 20 26  
 F: +90 312 434 17 67  
 info@medi-turk.com  
 www.medi-turk.com

medi UK Ltd.  
 Plough Lane  
 Hereford HR4 OEL  
 Great Britain  
 T +44 1432 37 35 00  
 F +44 1432 37 35 10  
 enquiries@mediuk.co.uk  
 www.mediuk.co.uk

medi Ukraine LLC  
 Evhena Sverstiuka str, 11.  
 Kiev 02002  
 Ukraine  
 T: +380 44 591 11 63  
 F: +380 44 392 73 73  
 info@medi.ua  
 http://medi.ua

medi USA L.P.  
 6481 Franz Warner Parkway  
 Whitsett, N.C. 27377-3000  
 USA  
 T +1-336 4 49 44 40  
 F +1-888 5 70 45 54  
 info@mediusa.com  
 www.mediusa.com

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**Important notes**

This medical device **MD** is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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Polyamid, Polyester, Elastan, Viskose

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# Genumedi®

## Intended purpose

Genumedi is a knee bandage for soft tissue compression.

## Indications

All indications which call for soft tissue compression with local pressure pad(s) on the kneecap, such as:

- Mild instability
- Osteoarthritis and arthritis
- Patellofemoral pain syndrome (runner's knee)

## Contraindications

No general side effects are known. However, if supports (or similar aids) are too tight, local pressure effects or constriction of blood vessels or nerves may occur in individuals. If you have any of the following complaints please consult your doctor before using the brace:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).
- Sensitivity and perfusion disorders of the legs or feet (e.g. in the case of diabetes, varicose veins).
- Lymph outflow disorders, as well as undiagnosed swellings of soft tissues, apart from the area of use.

Any combination with other products (e.g. compression stockings) must be discussed in advance with the responsible doctor.

## Intended patient groups

Healthcare professionals should provide care to the adults and children, applying the available information on the measurements/sizes and necessary functions/indications and in line with the information provided by the manufacturer, and acting under their own responsibility.

## Mode of Action / Comment on Wear

The integrated silicone ring of the Genumedi reduces stress on the kneecap. Together with the compression of the knitted fabric, it causes a massage effect. This enhances perfusion of the joint and decreases swelling and effusion. Irritation and pain are alleviated. At the same time, muscular control and inherent stabilisation of the knee joint are improved.

Supports are most beneficial during physical activity. The support can in principle be worn throughout the day. This must depend on how comfortable the wearer is. Moreover, the knee may not be bent by more than 70° when sitting. The support should be removed for long breaks (e.g. long periods sitting, long drives or sleeping).

The support is suitable for use in water. After use, please rinse thoroughly

## Putting on / Taking off

- Grasp the support with the thumb on the internal upper end of the lateral knob adhesive bands (Fig. 1)
- Pull the support over the knee in such a way that the silicone ring covers the kneecap centrally and free of pressure (Figs. 2 +3).
- To take off the support, grasp the lower lateral rods and pull it downwards and away (Fig. 4).

## Care instructions

The silicone ring is not supposed to be removed before washing. We recommend using a laundry bag. Fabric softener, fats, oils and ointments can attack the material and the Clima Comfort Effect can be impaired. Soap residues can cause skin irritation and material wear.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

**Storage instructions**

Keep the product in a cool, dry place and do not expose to direct sunlight.

**Material composition**

Polyamide, polyester, elastane, viscose

**Liability**

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

**Disposal**

The product can be disposed of in the domestic waste.

**Your medi Team**

Wishes you a rapid recovery!

Since made-to-measure garments are custom-made devices pursuant to EU Directive 93/42/EEC they do not bear any CE marketing. They satisfy the fundamental requirements of Attachment I of the Directive custom-made device.

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR). The traceability of this product is assured via a UDI code **UDI**.

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