medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-783
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61 2 9890 8696
 F +61 2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43-512 57 95 15
 F +43-512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium NV
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T: + 32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Canada Inc / médi Canada Inc
 597, Rue Duvernay, Verchères
 QC Canada J0L 2R0
 T: +1 450-583-3317 / +1 800 361 3153
 F: +1 888-583-6827
 service@medicanada.ca
 www.medicanada.ca

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45 70 25 56 10
 F +45 70 25 56 20
 kundeservice@sw.dk
 www.medit danmark.dk

medi Bayreuth España SL
 C/Canigó 2 – 6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34 93 260 04 00
 F +34 93 260 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33 1 48 61 76 10
 F +33 1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1371 0090
 F +36 1371 0091
 info@medi.hu
 www.medi.hu

medi Japan K.K.
 5-1 Nihonbashi Kabutocho Chuoku
 103-0026 Tokyo
 Japan
 T: +81 3 6778 2590
 F: +81 3 5847 7901
 info@medi-japan.jp
 www.medi-japan.co.jp

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31 76 57 22 555
 F +31 76 57 22 565
 info@medi.nl
 www.medi.nl

medi Norway AS
 Vestheimvegen 35
 4250 Kopervik
 Norway
 T: +47 52 84 45 00
 F: +47 52 84 45 09
 post@medinorway.no
 www.medinorway.no

medi Polska Sp. z o.o.
 ul. Łabędzka 22
 44-121 Gliwice
 Poland
 T: +48-32 230 60 21
 F: +48-32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal Lda.
 Rua do Centro Cultural, no. 43
 1700-106 Lisbon
 Portugal
 T +351 21 843 71 60
 F +351 21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

medi RUS LLC
 Business Center NEO GEO
 Butlerova Street 17
 117342 Moscow
 Russia
 T +7-495 374 04 56
 F +7-495 374 04 56
 info@medirus.ru
 www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
 Room 1703, T1, CES WEST BUND CENTER,
 No. 277, Long Lan Road
 Xuhui District
 200232 Shanghai
 People's Republic of China
 T: +86-21 50582319
 F: +86-21 50582319

medi Sweden AB
 Box 6034
 192 06 Sollentuna
 Sweden
 T +46 8 96 97 98
 F +46 8 626 68 70
 info@medi.se
 www.medi.se

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Important notes

This medical device **MD** is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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Bandage: Polyamid, Polyester, Elasthan, Viskose

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Lumbamed® plus

Intended purpose

Lumbamed plus is a brace for stabilising the lumbar spine using a back pad.

Indications

All indications which require stabilisation of the lumbar spine by redistributing pressure using a back pad, such as:

- Lower back pain (lumbago)
- Postural muscle imbalance in the lumbar spine

Contraindications

None known at present.

Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

Intended patient groups

Healthcare professionals should provide care to the adults and children, applying

the available information on the measurements/sizes and necessary functions/indications and in line with the information provided by the manufacturer, and acting under their own responsibility.

Note on wearing

Supports have the greatest effect during physical exercise. In principal, the support can be worn all day. The special cut in the area of the groin (Comfort Zone) is decisive for the correct and comfortable fit of the support. However, this depends on how comfortable the support is to wear. The support should be taken off for longer breaks (e.g. sitting for a long time or a long car drive, sleeping).

The support is also suitable for use in water. After use, please rinse out the product thoroughly.

Putting on / Taking off

- Pull the fabric cover over the massage pad and secure the pad in the centre or in accordance with the physician's or healthcare professional's instructions on the middle grey hook and loop section between the two rods. The pad should be secured in such a way that the bottom end rests against the top of the area where the two buttocks come together (Fig. 1+2).
- The label on the right inside of the fastener provides information about how to put the support on (Fig. 3).
- First, slip into the wrist strap in the left inside of the fastener (Fig. 4).
 - Wrap the bandage around the body and then please refer back to the label to check whether you are holding the bandage correctly (Fig. 5).
 - Now, slip into the outer wrist strap on the right side (Fig. 6).
- Pull the two fastening sides over each

other and then secure them as centrally as possible with the hook and loop fastener. The bandage has been correctly applied when the bottom edge of the fastener is located just above the pubic bone (Fig. 7+8).

- The small finger tab on the edge of the right side of the fastener will help you to undo the bandage again (Fig. 9).

The Clima Comfort knitted fabric will achieve its best breathing activity when the bandage is worn directly against the skin.

Care instructions

Do not wash the pad together with the support. It can be cleaned with a damp cloth. Hook and loop fasteners should be closed for washing. We recommend using a laundry bag. Fabric softener, fats, oils and ointments can attack the material and the Clima Comfort Effect can be impaired. Soap residues can cause skin irritation and material wear.

- You can wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

Composition of the Material

Support: Polyamide, polyester, elastane, viscose

Pad: Polyurethane, polyethylene

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

Your medi Team

Wishes you a rapid recovery!

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR). The traceability of this product is assured via a UDI code [UDI](#).

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