medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-783
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61-2 9890 8696
 F +61-2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium NV
 Staatsbaan 77/0099
 3945 Ham
 Belgium
 T: +32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Canada Inc / médi Canada Inc
 597, Rue Duvernay, Verchères
 QC Canada J0L 2R0
 T: +1 450-583-3317 / +1 800 361 3153
 F: +1 888-583-6827
 service@medicanada.ca
 www.medicanada.ca

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegårdsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45-70 25 56 10
 F +45-70 25 56 20
 kundeservice@sw.dk
 www.medit danmark.dk

medi Bayreuth Espana SL
 C/Canigo 2-6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34-932 60 04 00
 F +34-932 60 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33-1 48 61 76 10
 F +33-1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1 371-0090
 F +36 1 371-0091
 info@medi.hu
 www.medi.hu

medi Japan K.K.
 5-1 Nihonbashi Kabutocho Chuoku
 103-0026 Tokyo
 Japan
 T: +81 3 6778 2590
 F: +81 3 5847 7901
 info@medi-japan.jp
 www.medi-japan.co.jp

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31-76 57 22 555
 F +31-76 57 22 565
 info@medi.nl
 www.medi.nl

medi Polska Sp. z o.o.
 ul. Łabędzka 22
 44-121 Gliwice
 Poland
 T: +48-32 230 60 21
 F: +48-32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
 Rua do Centro Cultural, no. 43
 1700-106 Lisbon
 Portugal
 T +351-21 843 71 60
 F +351-21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

medi RUS LLC
 Business Center NEO GEO
 Butlerova Street 17
 117342 Moscow
 Russia
 T +7-495 374 04 56
 F +7-495 374 04 56
 info@medirus.ru
 www.medirus.ru

medi Sweden AB
 Box 6034
 192 06 Sollentuna
 Sweden
 T +46 8 96 97 98
 F +46 8 626 68 70
 info@medi.se
 www.medi.se

medi Trading (Shanghai) Co. Ltd.
 Room 1703, T1, CES WEST BUND CENTER,
 No. 277, Long Lan Road
 Xuhui District
 200232 Shanghai
 People's Republic of China
 T: +86-21 50582319
 F: +86-21 50582319

Medi Turk Ortopedi Medikal Ic ve Dis Tic. AS
 Mustafa Kemal Mah. 2156 Sokak No: 16 /4-5
 06510 Çankaya Ankara
 Turkey
 T: +90 312 435 20 26
 F: +90 312 434 17 67
 info@medi-turk.com
 www.medi-turk.com

medi UK Ltd.
 Plough Lane
 Hereford HR4 0EL
 Great Britain
 T +44-1432 37 35 00
 F +44-1432 37 35 10
 enquiries@mediuk.co.uk
 www.mediuk.co.uk

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Important notes

This medical device **MD** is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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Bandage: Polyamid, Polyester, Elasthan, Viskose

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Lumbamed[®] sacro

Intended purpose

Lumbamed sacro is a brace for stabilising the pelvis.

Indications

All indications which require stabilisation of the pelvis, such as:

- Sacroiliac joint dysfunction
- Sacroiliac joint instability
- Muscle pain and tendon disease in the pelvic region
- Pelvic ring instability
- For recurring muscle and tendon disease (rectus abdominis muscle, piriformis muscle)

Contraindications

None known at present.

Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

Intended patient groups

Healthcare professionals should provide care to the adults and children, applying the available information on the measurements/sizes and necessary functions/indications and in line with the information provided by the manufacturer, and acting under their own responsibility.

Note on wearing

The brace's effects are most noticeable during movement of the body. In principle, this brace can be worn for the whole day. The length of time that the brace is worn should, however, be based upon the individual's own requirements and the way that their body feels.

The brace can also be worn in water. Please wash this product thoroughly with clear water and observe the instructions for care.

Putting on / taking off

- Before you put the brace on, loosen the fastening straps so that the brace can be put on easily. Release the belts' fasteners and fix them loosely at the back of the belt, on the velour.
- Fix the pads in the middle of and/or according to the doctor's or specialist's instructions on the grey hook and loop. If no other advice has been given, the pads should be fixed with the narrow end pointing downwards (Fig. 1 & 2).
- The label on the right inside section provides instructions on how to put the brace on (Fig. 3).
- First, slip your hand into the hand strap on the left interior fastening side.
 - Loop the brace around your pelvis (Fig. 4).
 - Now, slip your hand into the outer hand strap on the right-hand side (Fig. 5).
- The brace should be fastened with the bottom edge of the fastener sitting

just above the pubis. Pull the two fastening sides over each other and fasten them as centrally as possible (Fig. 6).

- Then release the fastening straps' fasteners and pull these to the front simultaneously. You can adjust the tightness of the brace according to your needs and level of pain (Fig. 7 + 8).
- To check the position of the brace, lift your thigh up slightly. The bottom of the fastening strap should be level with your groin (Fig. 9).
- You can remove parts of the brace and just wear the outer, non-elastic component with the fastening strap system (Fig. 12). For this, please loosen the hook and loop fastenings on both sides of the fastening interior side and remove the elastic knitted fabric (Fig. 10).
- You can also place the pads in the centre of the belt here or position them here according to your doctor's or a specialist's instructions, if required (Fig. 11).
- Then put the brace on as described above (Fig. 12).

Our recommendation: Wear the brace directly over underwear, or a very thin layer of clothing, so that the massage pads can have an optimal effect.

If you also wish to wear this brace at night, please observe the additional instructions for use for the Fastening Extension.

There is no data on the use of Lumbamed sacro by pregnant women. Therefore, only doctors may decide whether this brace is appropriate, and doctors must be consulted for clarification in individual cases.

Instructions for care

The pads should be removed before washing. If necessary, the pads can be cleaned with a dampened cloth. Please fasten all hook and loop fastenings before washing. We recommend that you use a laundry bag. Fats, oils, lotions and balms can damage the material and impair the Clima Comfort effect. Soap residue can lead to inflammation of the skin and deterioration of the material.

- Wash the product by hand, preferably with mediclean soap, or in a delicate wash cycle at 30°C with mild detergent and without fabric conditioner.
- Allow to air-dry.
- Do not bleach.
- Do not iron.
- Do not dry-clean.

Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

Material composition

Band: Polyamide, polyester, elastane, viscose

Pads: Polyurethane

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in these instructions for use.

Disposal

You can dispose of this product with your household waste.

Your medi Team

wishes you a speedy recovery!

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR). The traceability of this product is assured via a UDI code UDI.

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