**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**D-95448 Bayreuth**  
**Germany**  
**T +49 921 912-0**  
**F +49 921 912-783**  
**ortho@medi.de**  
**www.medi.de**

medi Australia Pty Ltd  
 83 Fennell Street  
 North Parramatta NSW 2151  
 Australia  
 T +61-2 9890 8696  
 F +61-2 9890 8439  
 sales@mediaustralia.com.au  
 www.mediaustralia.com.au

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43 512 57 95 15  
 F +43 512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium NV  
 Staatsbaan 77/0099  
 3945 Ham  
 Belgium  
 T: + 32-11 24 25 60  
 F: +32-11 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

medi Canada Inc / médi Canada Inc  
 597, Rue Duvernay, Verchères  
 QC Canada J0L 2R0  
 T: +1 450-583-3317 / +1 800 361 3153  
 F: +1 888-583-6827  
 service@medicanada.ca  
 www.medicanada.ca

MAXIS a.s.,  
 medi group company  
 Slezská 2127/13  
 120 00 Prague 2  
 Czech Republic  
 T: +420 571 633 510  
 F: +420 571 616 271  
 info@maxis-medica.com  
 www.maxis-medica.com

medi Danmark ApS  
 Vejlegardsvej 59  
 2665 Vallensbaek Strand  
 Denmark  
 T +45-70 25 56 10  
 F +45-70 25 56 20  
 kundeservice@sw.dk  
 www.medit danmark.dk

medi Bayreuth Espana SL  
 C/Canigo 2-6 bajos  
 Hospitalet de Llobregat  
 08901 Barcelona  
 Spain  
 T +34-932 60 04 00  
 F +34-932 60 23 14  
 medi@mediespana.com  
 www.mediespana.com

medi France  
 Z.I. Charles de Gaulle  
 25, rue Henri Farman  
 93297 Tremblay en France Cedex  
 France  
 T +33-1 48 61 76 10  
 F +33-1 49 63 33 05  
 infos@medi-france.com  
 www.medi-france.com

medi Hungary Kft.  
 Bokor u. 21.  
 1037 Budapest  
 Hungary  
 T +36 1 371-0090  
 F +36 1 371-0091  
 info@medi.hu  
 www.medi.hu

medi Japan K.K.  
 5-1 Nihonbashi Kabutocho Chuoku  
 103-0026 Tokyo  
 Japan  
 T: +81 3 6778 2590  
 F: +81 3 5847 7901  
 info@medi-japan.jp  
 www.medi-japan.co.jp

medi Nederland BV  
 Heusing 5  
 4817 ZB Breda  
 The Netherlands  
 T +31-76 57 22 555  
 F +31-76 57 22 565  
 info@medi.nl  
 www.medi.nl

medi Polska Sp. z o.o.  
 ul. Łąbiedzka 22  
 44-121 Gliwice  
 Poland  
 T: +48-32 230 60 21  
 F: +48-32 202 87 56  
 info@medi-polska.pl  
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda  
 Rua do Centro Cultural, no. 43  
 1700-106 Lisbon  
 Portugal  
 T +351-21 843 71 60  
 F +351-21 847 08 33  
 medi.portugal@medibayreuth.pt  
 www.medi.pt

medi RUS LLC  
 Business Center NEO GEO  
 Butlerova Street 17  
 117342 Moscow  
 Russia  
 T +7-495 374 04 56  
 F +7-495 374 04 56  
 info@medirus.ru  
 www.medirus.ru

medi Sweden AB  
 Box 6034  
 192 06 Sollentuna  
 Sweden  
 T +46 8 96 97 98  
 F +46 8 626 68 70  
 info@medi.se  
 www.medi.se

medi Trading (Shanghai) Co. Ltd.  
 Room 1703, T1, CES WEST BUND CENTER,  
 No. 277, Long Lan Road  
 Xuhui District  
 200232 Shanghai  
 People's Republic of China  
 T: +86-21 50582319  
 F: +86-21 50582319

Medi Turk Ortopedi Medikal Ic ve Dis Tic. AS  
 Mustafa Kemal Mah. 2156 Sokak No: 16 /4-5  
 06510 Çankaya Ankara  
 Turkey  
 T: +90 312 435 20 26  
 F: +90 312 434 17 67  
 info@medi-turk.com  
 www.medi-turk.com

medi UK Ltd.  
 Plough Lane  
 Hereford HR4 0EL  
 Great Britain  
 T +44-1432 37 35 00  
 F +44-1432 37 35 10  
 enquiries@mediuk.co.uk  
 www.mediuk.co.uk

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### Important notes

This medical device **MD** is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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# Lumbamed<sup>®</sup> sacro Fastening Extension

## Intended purpose

The Fastening Extension for the Lumbamed sacro is available as an option, as an additional element. This product is to be used exclusively in conjunction with the Lumbamed sacro. The Fastening Extension is suitable for use with all available sizes of the Lumbamed sacro.

---

Further information on indications, contra-indications, risks / side-effects and notes on wearing can be found in the instructions for use for the Lumbamed sacro.

## Note on wearing

The 3-in-1 design of the Lumbamed sacro combines the advantages of a brace, bandage and pelvic band. The modular design of the Lumbamed sacro enables you to adjust the device to your levels of pain and activity. For comfortable support, for example during the night or when doing sport, we recommend that you use the knitted bandage. Here, the Fastening Extension is needed so that you can attach the bandage to your body securely, according to your needs.

## Putting on / taking off

- Firstly, undo the velcro fastenings on the interior fastening side of the Lumbamed sacro and remove the pelvic band (Fig. 1). You are now holding the bandage (Fig. 2). The label on the right inner side of the bandage

provides instructions on how to put the bandage on later.

- The Fastening Extension comes with two fastening parts. First, take the fastening part with the medi logo on it and open the velcro fastening (Fig. 3).
- Now put the right end of the bandage in the open end of the fastening and close it (Fig. 4).
- Take the second fastening and attach this to the left end of the bandage using the same process (Fig. 5). Ensure that the hand strap remains on the inner side. The bandage can now be put on (Fig. 6 + 7).
- To complete the brace again and connect it with the pelvic band, remove both parts of the Extension Fastening by undoing the velcro. Then attach the knitted bandage to the non-elasticated pelvic band using the velcro in the middle of these components. Ensure that the coloured fastening edge and the medi logo on the back are both on the bottom of the device. Finally, reattach both ends of the bandage to the inside of the pelvic band using the velcro fastenings (Fig. 1).

---

## Care instructions

The fastening can be washed with the bandage or on its own. Please close all velcro fastenings and stick fastening parts to each other before washing. We recommend using a laundry bag. Fabric softeners, fats, oils, lotions, ointments and soap residues can affect the material, hinder the climate comfort effect and cause skin irritations.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.

- Do not iron.
- Do not dry clean.

### Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

### Material composition

Polyamide, poliurethane

### Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

### Disposal

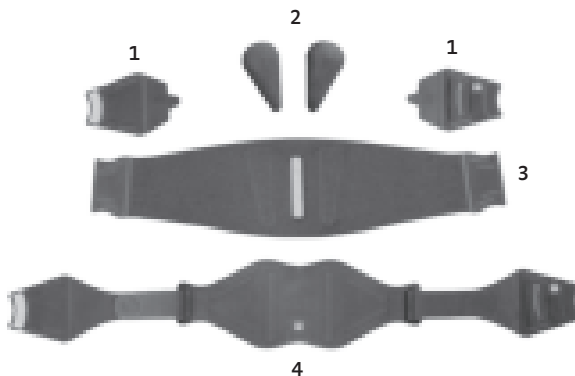
The product can be disposed of in the domestic waste.

### Your medi team

Wishes you a speedy recovery!

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR). The traceability of this product is assured via a UDI code [UDI](#).

1. Fastening Extension
2. Massage pads
3. Knitted bandages
4. Non-elasticated fastening strap system (pelvic band)

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