medi GmbH & Co. KG  
Medicusstraße 1  
95448 Bayreuth  
Germany  
T +49 921 912-0  
F +49 921 912-780  
ortho@medi.de  
www.medi.de

medi Australia Pty Ltd  
83 Fennell Street  
North Parramatta NSW 2151  
Australia  
T +61-2 9890 8696  
F +61-2 9890 8439  
sales@mediaustralia.com.au  
www.mediaustralia.com.au

medi Austria GmbH  
Adamgasse 16/7  
6020 Innsbruck  
Austria  
T +43 512 57 95 15  
F +43 512 57 95 15 45  
vertrieb@medi-austria.at  
www.medi-austria.at

medi Belgium NV  
Staatsbaan 77/0099  
3945 Ham  
Belgium  
T: +32-11 24 25 60  
F: +32-11 24 25 64  
info@medi-belgium.be  
www.medi-belgium.be

medi Canada Inc / médi Canada Inc  
597, Rue Duvernay, Verchères  
QC Canada J0L 2R0  
T: +1 450-583-3317 / +1 800 361 3153  
F: +1 888-583-6827  
service@medicanada.ca  
www.medicanada.ca

MAXIS a.s.,  
medi group company  
Slezská 2127/13  
120 00 Prague 2  
Czech Republic  
T: +420 571 633 510  
F: +420 571 616 271  
info@maxis-medica.com  
www.maxis-medica.com

medi Danmark ApS  
Vejlegårdsvej 59  
2665 Vallensbæk Strand  
Denmark  
T +45-70 25 56 10  
F +45-70 25 56 20  
kundeservice@sw.dk  
www.meditandmark.dk

medi Bayreuth Espana SL  
C/Canigo 2-6 bajos  
Hospital de Llobregat  
08901 Barcelona  
Spain  
T +34-932 60 04 00  
F +34-932 60 23 14  
medi@mediespana.com  
www.mediespana.com

medi France  
Z.I. Charles de Gaulle  
25, rue Henri Farman  
93297 Tremblay en France Cedex  
France  
T +33-1 48 61 76 10  
F +33-1 49 63 33 05  
infos@medi-france.com  
www.medi-france.com

medi Hungary Kft.  
Bokor u. 21.  
1037 Budapest  
Hungary  
T +36 1 371-0090  
F +36 1 371-0091  
info@medi.hu  
www.medi.hu

medi Japan K.K.  
5-1 Nihonbashi Kabutocho Chuoku  
103-0026 Tokyo  
Japan  
T: +81 3 6778 2590  
F: +81 3 5847 7901  
info@medi-japan.jp  
www.medi-japan.co.jp

medi Nederland BV  
Heusing 5  
4817 ZB Breda  
The Netherlands  
T +31-76 57 22 555  
F +31-76 57 22 565  
info@medi.nl  
www.medi.nl

medi

# Manumed

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### Important notes

This medical device **MD** is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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medi Trading (Shanghai) Co. Ltd.  
Room 1703, T1, CES WEST BUND CENTER,  
No. 277, Long Lan Road  
Xuhui District  
200232 Shanghai  
People's Republic of China  
T: +86-21 50582319  
F: +86-21 50582319

Medi Turk Ortopedi Medikal İc ve Dis Tic. AS  
Mustafa Kemal Mah. 2156 Sokak No: 16/4-5  
06510 Çankaya Ankara  
Turkey  
T: +90 312 435 20 26  
F: +90 312 434 17 67  
info@medi-turk.com  
www.medi-turk.com

medi UK Ltd.  
Plough Lane  
Hereford HR4 0EL  
Great Britain  
T +44-1432 37 35 00  
F +44-1432 37 35 10  
enquiries@mediuk.co.uk  
www.mediuk.co.uk

medi Ukraine LLC  
Evhena Sverstiuka str, 11.  
Kiev 02002  
Ukraine  
T: +380 44 591 11 63  
F: +380 44 392 73 73  
info@medi.ua  
http://medi.ua

medi USA L.P.  
6481 Franz Warner Parkway  
Whitsett, N.C. 27377-3000  
USA  
T +1-336 4 49 44 40  
F +1-888 5 70 45 54  
info@mediusa.com  
www.mediusa.com

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# Manumed

## Intended purpose

Manumed is a wrist brace for immobilizing the wrist in at least two directions of mobilisation.

---

## Indications

All indications which require the wrist to be immobilised for at least two directions of mobilisation whilst maintaining free finger movement, such as:

- Immobilisation for sprains
- Tendonitis, tendosynovitis
- After distal radius fracture (nonsurgical, postoperative)
- Rheumatoid arthritis
- Carpal tunnel syndrome
- Postoperative after triangular fibrocartilage complex repair
- Guyon's canal syndrome

## Contraindications

Unstable fractures in the hand

## Risks / Side Effects

If the aids are too tight, this can cause pressure sores or constriction of blood vessels or nerves. Consult your doctor before using this device if you have the following conditions:

- A condition affecting the application area
- Paraesthesia, circulatory disorders
- Lymph drainage disorders

Tightly fitting aids can cause skin irritation (pressure, sweat, material).

## Intended patient groups

Healthcare professionals should provide care to the adults and children, applying the available information on the

measurements/sizes and necessary functions/indications and in line with the information provided by the manufacturer, and acting under their own responsibility.

### Applying the support

- The pre-shaped aluminium splint can be removed and easily moulded to match different wrist contours (fig. 1).
- Open all the straps and slip your hand into the support (with the aluminium splint at the bottom – fig. 2).
- First close the front strap and then the back strap (fig. 3).
- Now close the broad central strap. This strap is attached to the underside of the support by hooks and loops and can be moved before fastening if necessary (fig. 4 + 5).

---

### Care instructions

Please close the hook and loop fasteners before washing and remove the orthoses. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using medicated washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

---

### Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

---

### Material composition

Aluminum, polyamide.

**Liability**

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

**Disposal**

The product can be disposed of in the domestic waste.

**Your medi team**

wishes you a quick return to full fitness.

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR). The traceability of this product is assured via a UDI code UDI.

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