**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**D-95448 Bayreuth**  
**Germany**  
**T +49 921 912-0**  
**F +49 921 912-780**  
**ortho@medi.de**  
**www.medi.de**

medi Australia Pty Ltd  
 83 Fennell Street  
 North Parramatta NSW 2151  
 Australia  
 T +61-2 9890 8696  
 F +61-2 9890 8439  
 sales@mediaustralia.com.au  
 www.mediaustralia.com.au

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43 512 57 95 15  
 F +43 512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium NV  
 Staatsbaan 77/0099  
 3945 Ham  
 Belgium  
 T: +32-11 24 25 60  
 F: +32-11 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

medi Canada Inc / médi Canada Inc  
 597, Rue Duvernay, Verchères  
 QC Canada J0L 2R0  
 T: +1 450-583-3317 / +1 800 361 3153  
 F: +1 888-583-6827  
 service@medicanada.ca  
 www.medicanada.ca

MAXIS a.s.,  
 medi group company  
 Slezská 2127/13  
 120 00 Prague 2  
 Czech Republic  
 T: +420 571 633 510  
 F: +420 571 616 271  
 info@maxis-medica.com  
 www.maxis-medica.com

medi Danmark ApS  
 Vejlegardsvej 59  
 2665 Vallensbaek Strand  
 Denmark  
 T +45-70 25 56 10  
 F +45-70 25 56 20  
 kundeservice@sw.dk  
 www.medit danmark.dk

medi Bayreuth Espana SL  
 C/Canigo 2-6 bajos  
 Hospitalet de Llobregat  
 08901 Barcelona  
 Spain  
 T +34-932 60 04 00  
 F +34-932 60 23 14  
 medi@mediespana.com  
 www.mediespana.com

medi France  
 Z.I. Charles de Gaulle  
 25, rue Henri Farman  
 93297 Tremblay en France Cedex  
 France  
 T +33-1 48 61 76 10  
 F +33-1 49 63 33 05  
 infos@medi-france.com  
 www.medi-france.com

medi Hungary Kft.  
 Bokor u. 21.  
 1037 Budapest  
 Hungary  
 T +36 1 371-0090  
 F +36 1 371-0091  
 info@medi.hu  
 www.medi.hu

medi Japan K.K.  
 5-1 Nihonbashi Kabutocho Chuoku  
 103-0026 Tokyo  
 Japan  
 T: +81 3 6778 2590  
 F: +81 3 5847 7901  
 info@medi-japan.jp  
 www.medi-japan.co.jp

medi Nederland BV  
 Heusing 5  
 4817 ZB Breda  
 The Netherlands  
 T +31-76 57 22 555  
 F +31-76 57 22 565  
 info@medi.nl  
 www.medi.nl

medi Polska Sp. z o.o.  
 ul. Łąbiedzka 22  
 44-121 Gliwice  
 Poland  
 T: +48-32 230 60 21  
 F: +48-32 202 87 56  
 info@medi-polska.pl  
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda  
 Rua do Centro Cultural, no. 43  
 1700-106 Lisbon  
 Portugal  
 T +351-21 843 71 60  
 F +351-21 847 08 33  
 medi.portugal@medibayreuth.pt  
 www.medi.pt

medi RUS LLC  
 Business Center NEO GEO  
 Butlerova Street 17  
 117342 Moscow  
 Russia  
 T +7-495 374 04 56  
 F +7-495 374 04 56  
 info@medirus.ru  
 www.medirus.ru

medi Sweden AB  
 Box 6034  
 192 06 Sollentuna  
 Sweden  
 T +46 8 96 97 98  
 F +46 8 626 68 70  
 info@medi.se  
 www.medi.se

medi Trading (Shanghai) Co. Ltd.  
 Room 1703, T1, CES WEST BUND CENTER,  
 No. 277, Long Lan Road  
 Xuhui District  
 200232 Shanghai  
 People's Republic of China  
 T: +86-21 50582319  
 F: +86-21 50582319

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
Medi Turk Ortopedi Medikal  
 Ic ve Dis Tic. AS  
 Mustafa Kemal Mah. 2156 Sokak  
 No: 16 /4-5  
 06510 Çankaya Ankara  
 Turkey  
 T: +90 312 435 20 26  
 F: +90 312 434 17 67  
 info@medi-turk.com  
 www.medi-turk.com

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#### Important notes

This medical device **MD** is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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# medi SAS® 45

## Intended purpose

The medi SAS 45 is a shoulder joint support for immobilisation and stress relief.

## Indications

For all indications which require immobilisation and stress relief of the shoulder joint in abduction, such as:

- After rotator cuff repair
- Humeral head fractures (nonsurgical, postoperative)
- After insertion of shoulder prosthesis
- After stabilization of acromioclavicular joint dislocations
- After biceps tenodesis
- Impingement syndrome (nonsurgical, postoperative)

## Contraindications

All indications which expressly require internal rotation or abduction of more than 45° or less than 30°.

## Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical

irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

## Intended patient groups

Healthcare professionals should provide care to the adults and children, applying the available information on the measurements/sizes and necessary functions/indications and in line with the information provided by the manufacturer, and acting under their own responsibility.

## Important notes

Only change the angle of abduction on the instructions of your doctor.

## Fitting instructions

the medi SAS 45 is supplied in a single universal size and has two options for use, providing either 45° or 30° of abduction.

- Loosen all the straps including the hand section.
- Place the cushion on the hip of the affected side and lay the shoulder strap over the opposite shoulder. Fasten the straps to the cushion with the strip, first the back and then the front. Attach one end of the hip belt to the front of the cushion.
- Now secure the hand rest to the top of the cushion with the hook and loop fastener.
- Attach the two forearm straps to the hand rest, and secure the upper arm strap to the cushion slightly obliquely. The upper arm strap prevents the arm slipping backwards.
- Slip your arm into the sling and fasten all the strips. Make sure that the straps are not fastened too tightly. Simply rotate the cushion to select 45° or 30° abduction.

Note: Make sure that the head of the humerus is not pushed upwards when the upper arm is resting on the cushion.

---

### Care instructions

The product should not be worn in water. Please close hook and loop fasteners prior to washing. Traces of soap, creams or ointments can cause skin irritations and material wear.

- Pillow case, bag and straps can be washed by hand, preferably using mediclean washing agent, or in the washing machine with the delicate cycle at 30°C using mild detergent without fabric softener.
  - Do not bleach.
  - Air dry.
  - Do not iron.
  - Do not dry clean.
- 

### Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

---

### Material composition

PU-foam, Polyamid

### Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

### Disposal

The product can be disposed of in the domestic waste.

### Your medi team

wishes you a quick return to full fitness.

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR). The traceability of this product is assured via a UDI code [UDI](#).

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