Maxis®

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Nevratné

Dátum revízie: 12/2020

Dear user,
thank you for your trust. We are pleased that you have opted for our medical compression arm sleeves or compression gloves. To ensure the effectiveness of your compression therapy, we ask that you take some time to read these instructions carefully. Keep them in a safe place so that you can find important information later if needed. If you have any questions, please contact your doctor or healthcare staff you trust.

1. Purpose – what are medical compression arm sleeves or compression gloves used for?

Medical compression arm sleeves or compression gloves are an integral part of compression therapy. Compression therapy is part of the treatment for venous, lymphatic and mixed edema and is one of the irreplaceable methods of conservative care.

- postoperative edema
- postoperative reperfusion edema
- cyclic idiopathic edema
- lipedema
- congestion due to immobility (paresis and partial limb paresis)
- drug-induced edema, if switching to another drug is not possible

2. Performance characteristics – how do medical compression arm sleeves or compression gloves work?

The mechanism of action is to reduce capillary filtration into the interstitium, assist in the resorption of tissue fluid (and crystalloids) into the venous capillary arm, accelerate lymph flow through the lymphatic system, and enhance the venous-lymphatic-muscle pump effect. This spectrum of effects minimizes or completely eliminates the adverse effects of venous reflux, venostasis and lymphostasis.

Other indications

- post-burn condition
- treatment of scars

Please note the following information:

Patient-specific factors such as body weight, type and severity of swelling, and the nature of connective tissue also play a role in the selection of medical compression arm sleeves or compression gloves.

3. Indications – what symptoms are medical compression arm sleeves or compression gloves used for?

After diagnosis you have received our medical compression device.

For optimal treatment results, follow the instructions of your doctor or healthcare staff. Please note that each compression arm sleeve or compression glove has different indications and areas of use due to the nature of the material and the available compression classes.

Therefore, our recommendations:

- The higher your body weight,
 - The greater the tendency of swelling,
 - The more severe the disease,
 - The softer the connective tissue,
- the thicker the sleeve material should be!

Your doctor or medical staff you trust will help and advise you on the correct choice of medical compression arm sleeves or compression gloves. This will ensure that your individual needs are taken into account – for greater wellbeing and therapeutic success.

Your medical compression arm sleeves or compression gloves can generally be used for the following indications:

Chronic venous diseases

- venous malformations

Thromboembolic venous diseases

- superficial venous thrombosis
- upper limb vein thrombosis
- condition after thrombosis
- post-thrombotic syndrome

Edema

- lymphedema
- post-traumatic edema

4. Contraindications – when should medical compression arm sleeves or compression gloves not be worn?

Medical compression arm sleeves or compression gloves must not be worn in the following cases:

- advanced peripheral arterial obliteration
- decompensated cardiac insufficiency (NYHA III + IV)
- septic phlebitis
- phlegmasia coerulea dolens

In the following cases, the therapeutic decision should be carefully considered (in view of the benefits and risks and the choice of the most appropriate compression):

- severe wet dermatosis

- intolerance to compression material
- severe limb sensitivity disorders
- advanced peripheral neuropathy (e.g. in diabetes mellitus)
- primarily chronic polyarthritis

If you are not sure if any of these apply to you, talk to your doctor or a healthcare professional you trust.

The following risks and side effects may occur:

In exceptional cases, medical compression arm sleeves or compression gloves can cause, especially in the case of improper handling

- skin necrosis and
 - pressure damage of peripheral nerves.
- Sensitive skin may be prone to itchiness, scaling and signs of inflammation. Therefore, it is reasonable to take care of your skin during compression care. Please follow our important guidelines (points 7 and 8).

The following symptoms should result in the immediate removal of the compression sleeve and consultation with the attending physician or a healthcare professional:

- blue or white fingers, discomfort and numbness, increasing pain, shortness of breath and sweating, acute movement restriction

5. Intended target user group

The intended users include doctors, healthcare professionals, nursing staff ensuring correct application of the medical compression sleeve to patients, and the patients themselves.

The correct choice of medical compression sleeve (measurement, size determination, function, indication in adults and children, compression class) is their responsibility and should be based on available information from the manufacturer.

6. Wearing and use time – how long can I wear compression arm sleeves or compression gloves?

Unless your doctor specifies otherwise, wear your compression arm sleeves or compression gloves from morning to evening for an optimal therapeutic effect.

For the arm sleeves or compression gloves to work optimally, a precisely graduated pressure curve is a prerequisite. Everyday wearing and washing can reduce the necessary therapeutic pressure and elasticity

of your compression arm sleeves or compression gloves over time. The recommended usability is therefore 6 months. To obtain the best pressure, follow our guidelines and recommendations (points 8 and 10). Do not hesitate to contact us. We will inform you about the possibilities of ensuring effective and long-term treatment of diseases of the venous or lymphatic system using medical compression arm sleeves or compression gloves.

7. Instructions for use – what should I know?

- Medical compression arm sleeves or compression gloves should only be distributed by trained personnel and accompanied by instructions for use and care (see points 8 and 10).
- Should you experience pain or excessive skin irritation during wear, remove your arm sleeves or compression gloves immediately and consult your doctor or a healthcare professional.
- Appropriate skin care is essential for compression therapy. In particular, creams or ointments containing fat, but also soap residues, can cause skin irritation and material wear, and thus reduce the effectiveness of medical compression arm sleeves or compression gloves. That's why we offer complementary skin care products that are specially adapted for compression arm sleeves or compression gloves. Ask for advice at specialized points of sale.
- In silicone banded compression arm sleeves or compression gloves, skin irritation may occur in people with sensitive skin (due to a combination of sweat and mechanical stress). To avoid this, it is recommended to slightly move the band several times during the day (see point 6). The slipping of arm sleeves or compression gloves can also be caused by a large amount of hairs. In similar cases, we recommend the use of a special adhesive for a better grip of the medical compression arm sleeves or compression gloves. If necessary, ask for advice at specialized points of sale.
- Sharp fingernails, jewelry, as well as rough and calloused palms can cause damage to the knit.
- Never repair arm sleeves or compression gloves by yourself or have them repaired by another service provider, as this reduces the quality, safety and efficiency of the medical device. Information on repair, if possible, will be provided to you by our salespeople or our customer service.
- In case of complaints about products such as knit damage, please contact your speci-

alized shops. Only serious events that may lead to a significant deterioration of health or death shall be reported to the manufacturer and the competent state authority. Major events are defined in Article 2 no. 65 of Regulation (EU) 2017/745 (MDR).

The textile label sewn onto the arm sleeves or compression gloves plays an important part in identifying the product. Under no circumstances should you remove this label from your medical compression arm sleeve or compression glove. Cutting it will void your warranty, repair, or replacement.

8. Donning instructions – what should I be careful about when putting on my medical compression arm sleeves or compression gloves?

- Always don the product on a dry limb, preferably in the morning before putting pressure on the limb.
- If you take a shower in the morning and then take care of your skin (see also point 10 – Instructions for care), wait a short time until your skin dries properly. This makes donning much easier. Also have textile or rubber gloves ready, which are available at specialized points of sale. You can use them later to make it easier to spread the sleeve around your hand or to get started at the beginning. The gloves fit particularly well and make donning easier.
- When donning, make sure your nails are rounded and smooth. Well-kept palms are also important to prevent damage to the knit. Put on your jewelry or wristwatch only after putting on your arm sleeves or compression gloves.

Donning medical compression arm sleeves or compression gloves

1. Gather the material of the sleeve up to the wrist. Insert your limb into the sleeve.
2. Begin stretching the sleeve around the limb. In order to prevent damage and tearing, do not just pull the upper edge of the sleeve, but stretch the sleeve gradually.
3. Spread the sleeve and smooth it out evenly over the entire upper limb.

Initially, you may find donning difficult, but with repeated use you will gain the skill you need and the process will become easier for you.

Tip: Ask for a practical metal donning aid – a medical compression upper limb donning aid.

9. Storage guidelines – how and how long can I store my medical compression arm sleeves or compression gloves?

The products should be stored in a dry place and protected from direct sunlight and heat.

The hourglass symbol on the box label indicates the latest usability date for the product. This is the maximum shelf life of serial products in the length of 42 months. This period already includes a maximum practical usability of 6 months (see point 6).

- Made-to-measure products are designed for immediate use.

10. Care guidelines – what do I have to look out for when caring for my medical compression arm sleeves or compression gloves?

Wash your arm sleeves or compression gloves every day after wearing. We recommend handwashing detergent for gentle care. Alternatively, use a mild laundry detergent (delicate wash at 40 °C) without an optical brightener and softener.

Tip:

- We recommend turning your compression arm sleeves or compression gloves inside out when washing. Wash them separately or with clothes of the same color.
- A washing bag better protects the sleeve material.
- Do not dry clean medical compression arm sleeves or compression gloves or wash them using household detergents.
- After washing, you can roll the arm sleeves or compression gloves into a towel and squeeze. Do not leave wet arm sleeves or compression gloves in a wet towel, air dry them. Under no circumstances should they be dried in the sun or on a radiator. Alternatively, you can delicately tumble dry your medical compression arm sleeves or compression gloves.

Wash up to 40° C (delicates)

Do not bleach

Low tumble dry temperatures (delicates)

Do not iron

Do not dry clean

11. Material composition – what are your compression arm sleeves or compression gloves composed of?

Compression arm sleeves or compression gloves MICRO contain polyamide and elastane. More detailed information can be found on the textile label sewn on your compression arm sleeves or compression gloves.

Without latex: There is no rubber from the milk juice of tropical plants processed in this product.

- MICRO Compression arm sleeves or compression gloves contain MICRO PROTECTION. It is an antibacterial and antifungal treatment that prevents the growth of fungi, bacteria and other undesirable microorganisms.

12. Disposal – how do I dispose of my medical compression arm sleeves or compression gloves?

Please dispose of your medical compression arm sleeves or compression gloves in your normal household waste. There are no special criteria for disposal.

13. Explanation of symbols

Unique Device Identifier

Unique Device Identifier

Single patient – multiple use (see also point 6): Wearing time and usability)

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