medi

protect.Heel soft / protect.Heel wedge

Fersenkissen aus Silikon · Silicone
heel cushion

Gebrauchsanweisung. Instructions for use. Mode
d'emploi. Instrucciones de uso. Instruções para
aplicação. Istruzioni per l'uso. Gebruiksaanwijzing.
Brugsvejledning. Bruksanvisning. Návod k použití.
Uputa za uporabu. Інструкція по використанню.
Kullanma kılavuzu. Instrukcja zakładania. Használati
útmutató. Інструкція з використання. 使用说明. 使用
說明書. 取扱説明書. Instrucțiuni de utilizare. دليل
الاستخدام. Інструкції за употреба. הוראות שימוש.

medi. I feel better.

protect.Heel soft / protect.Heel wedge

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protect.Heel soft / protect.Heel wedge

Intended use

Medical products for medical footcare for the patient which is in line with the indications.

Indications:

- The product serves to cushion the foot in various anatomical and functional areas to treat complaints, injuries and pain of various aetiologies in the forefoot, midfoot and hindfoot.

Contra-indications

- Currently unknown. If necessary, the product should not be used in direct contact with the skin if the user has a known silicone allergy. Equally, the product may not be worn on open wounds on the foot. If the patient presents with an unsteady gait, the product should be implemented under close observation by the therapeutic team. In the event of clear signs of critical gait characteristics, the products being evaluated should cease to be used in these circumstances, in order to prevent the patient from falling.

Important Notes

- This medical product is to be used exclusively for orthotic treatment of the foot and is only suitable for use on unbroken skin. It may not be worn on open wounds.
- This medical product is intended for multiple use by one individual patient.
- The medical product must be issued by experts or trained personnel.
- The stress that the product is placed under due to general wearing, salves, skin moisture and environmental influences can reduce its service life and impair the medical effectiveness of the product. Therefore, we recommend that your doctor and/or specialist retailer checks the product regularly.
- It is not permitted to make any changes to the product.

Intended users

- Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device,

English

provided that they have received briefing on this from healthcare professionals.

Patient target group

- Healthcare professionals should use this product to provide care to adults, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

Side effects

- No side effects that affect the entire organism are as yet known of, provided that the product is used properly. Sometimes, pressure points can be created or the wedge can slip in the shoe. Therefore always wear well-fitting shoes, otherwise using the wedge will incur risks.
- If your complaints worsen while wearing this aid, or if you notice negative changes in the area of the body that you are using it on (e.g. skin irritation), please report this to your doctor or medical specialist medical retailer immediately.
- If you know that you are intolerant to one of the materials in this aid, please discuss this with your doctor before using it.

How to wear the product (instructions for putting the product on)

- Note: Insofar as your doctor has not prescribed another course of action, always wear the protect.Heel soft / wedge heel wedges as a pair. This does not apply when the wedges are used to correct the difference if the patient's legs are different lengths. The protect.Heel soft / wedge heel wedges are only suitable for use if the skin is intact. They may not be worn on open wounds. Furthermore, they are intended for multiple use by one individual patient.
- Place your heel wedges in appropriate shoes. Ensure that the inside of the heel of the shoe fully encloses the heel wedge.
- If pain, skin irritation and/or discomfort occur whilst wearing the product, please remove the medical product immediately and contact your doctor or your specialist medical retailer.
- If you have experienced ongoing problems due to wearing this aid, please consult your doctor.

Instructions for care

- Wash by hand in lukewarm water using a mild detergent.
- Allow to dry completely.

- Do not dry this product on heating systems or other sources of direct heat.

Guarantee / warranty

- This medical product is intended to be used multiple times by one patient and is therefore not intended to be transferred to third parties. If the insole is used to treat more than one person or is altered in an incorrect manner, the product liability will be rendered null and void.
- No claims on the guarantee may be asserted for any damages which are caused by improper usage.
- Discolouration of the product does not constitute the grounds for a complaint.
- In the event of any complaints regarding the product such as damage to the aid or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

Transportation and storage conditions

- Store the product in a dry place, out of direct sunlight and away from heat.
- This aid has a shelf life of 42 months. This includes the storage life and the length of time that they may be used for. The length of time that they may be used for extends between 6 - 12 months, depending on how much and how heavily they are worn.

Disposal

- You can dispose of this product with your household waste.

Material composition

- 100% medical grade silicone

English

The date of the last revision of the instructions is stated on the last page of the instructions. Information on the manufacturer, the conformity symbol and, if required, the authorised representatives, is stated on the packaging.

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medi Canada Inc / médi Canada Inc
Medizintechnik GmbH & Co. KG