medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61 2 9890 8696
 F +61 2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43-512 57 95 15
 F +43-512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium NV
 Staatsbaan 77/0099
 3945 Ham
 Belgium
 T: + 32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Canada Inc / médi Canada Inc
 597, Rue Duvernay, Verchères
 QC Canada J0L 2R0
 T: +1 450-583-3317 / +1 800 361 3153
 F: +1 888-583-6827
 service@medicanada.ca
 www.medicanada.ca

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegårdsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45 70 25 56 10
 F +45 70 25 56 20
 kundeservice@sw.dk
 www.medit danmark.dk

medi Bayreuth España SL
 C/Canigó 2 – 6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34 93 260 04 00
 F +34 93 260 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33 1 48 61 76 10
 F +33 1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1371 0090
 F +36 1371 0091
 info@medi.hu
 www.medi.hu

medi Italia s.r.l.
 Via Giuseppe Ghedini, 2
 40069 Zola Predosa (BO)
 Italy
 T: +39-051 6 13 24 84
 F: +39-051 6 13 29 56
 info@medi-italia.it
 www.medi-italia.it

medi Japan K.K.
 5-1 Nihonbashi Kabutocho Chuoku
 103-0026 Tokyo
 Japan
 T: +81 3 6778 2590
 F: +81 3 5847 7901
 info@medi-japan.jp
 www.medi-japan.co.jp

Tachezy Sanit Srl.
 Via Druso 48/a
 39100 Bolzano
 Italy
 T: +39 0471 288 068
 F: +39 0471 282 773
 info@tachezysanit.com
 www.tachezysanit.com

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31 76 57 22 555
 F +31 76 57 22 565
 info@medi.nl
 www.medi.nl

medi Norway AS
 Vestheimvegen 35
 4250 Kopervik
 Norway
 T: +47 52 84 45 00
 F: +47 52 84 45 09
 post@medinorway.no
 www.medinorway.no

medi Polska Sp. z o.o.
 ul. Łabędzka 22
 44-121 Gliwice
 Poland
 T: +48-32 230 60 21
 F: +48-32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal Lda.
 Rua do Centro Cultural, no. 43
 1700-106 Lisbon
 Portugal
 T +351 21 843 71 60
 F +351 21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

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Important notes

This medical device **MD** is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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medi Trading (Shanghai) Co. Ltd.
Room 1703, T1, CES WEST BUND
CENTER,
No. 277, Long Lan Road
Xuhui District
200232 Shanghai
People's Republic of China
T: +86-21 50582319
F: +86-21 50582319

medi Sweden AB
Box 6034
192 06 Sollentuna
Sweden
T +46 8 96 97 98
F +46 8 626 68 70
info@medi.se
www.medi.se

Medi Turk Ortopedi Medikal Iç ve Dis Tic.
AS
Mustafa Kemal Mah. 2156 Sokak No: 16
/4-5
06510 Çankaya Ankara
Turkey
T: +90 312 435 20 26
F: +90 312 434 17 67
info@medi-turk.com
www.medi-turk.com

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
Great Britain
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Evhena Sverstiuka str, 11.
Kiev 02002
Ukraine
T: +380 44 591 11 63
F: +380 44 392 73 73
info@medi.ua
http://medi.ua

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Polyamid, Elastan, PU, Aluminium, Polyester

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protect.St

Intended purpose

protect.St II is a knee support brace with limitation of extension / flexion.

Indications

All indications that require reinforcement of the physiological stabilisation of the knee joint and/or stress relief of the joint, such as:

- Following injury to the collateral ligament
- For knee instability
- Following meniscal injury
- For knee osteoarthritis

Contraindications

None known at present.

Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

Intended patient groups

Healthcare professionals should provide

care to the adults and children, applying the available information on the measurements/sizes and necessary functions/indications and in line with the information provided by the manufacturer, and acting under their own responsibility.

Setting the flexion / extension limits

Extension limit wedge (limits extension):
0°, 10°, 20°, 30°, 45°

Flexion limit wedge (limits flexion):
0°, 10°, 20°, 30°, 45°, 60°, 75°, 90°

Can be immobilised at:

0°, 10°, 20°, 30°, 45°

The medial (inside) and the lateral (outside) wedge settings must always be the same.

1. 10° extension wedges are preinstalled in your knee brace as standard. They are found in the front aspect of the brace's hinge joint.
 2. To change the silver extension wedge loosen the screw and pull the wedge out from the front of the joint. You will find the replacement wedge in the wedge set. It is marked with the corresponding wedge setting. Slide the new extension wedge into the front of the joint. Now replace the screw in the threaded hole and screw it tight.
 3. The flexion wedges are installed from the back of the joint. Select the suitable wedge from the wedge set. Insert the wedge and secure it with the appropriate screw (enclosed).
 4. Bend and flex the brace 'joint' a couple of times and check that the wedges are positioned correctly. Finally, secure the joint cover.
-

Important notes

Only change the extension and flexion limits on instruction from your doctor or orthotist to do so. In order to prevent hyperextension, always insert extension

wedges or 0° wedges. The wedges define the exact excursion of the orthosis. In difficult soft-tissue conditions, e.g. where the soft-tissues are under compression, it may be necessary to insert the larger wedge to limit the joint movement to the desired excursion. In addition, make sure that the orthosis joints are correctly positioned: the middles of the joints should be level with the upper border of the patella. The joint must lie behind the midline of the leg.

Fitting instructions

- Open all straps and the two hook and loop fasteners.
- Now put your leg into a 45-degree angle flexion position. Wrap the orthosis tightly around your leg from the back and close it with the two hook and loop strips (Image 1),
- Make sure that the kneecap is in the middle of the patella recess.
- The centres of the brace joints should be at the top edge of the kneecap and just behind the lateral centre line of the leg (image 2).
- Now close the bottom strap and then the top strap (image 3).
- If possible, walk a few steps and check if the brace is properly fitted (image 4). Correct the fit of the brace at the leg, if required.

Care instructions

Please close the hook and loop fasteners before washing and remove the orthoses. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the brace in a dry place and do not expose to direct sunlight.

Material composition

Polyamide, elastane, PU, aluminum, polyester

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR). The traceability of this product is assured via a UDI code [UDI](#).

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