**RELAX**

**PREMIUM**

COMPRESSION STOCKINGS

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Dátum revízie návodu 09/2022

### Instructions for Use

The instructions apply to the following version: RELAX PREMIUM

Dear user,

thank you for your trust. We are pleased that you have opted for our medical compression stockings. To ensure the effectiveness of your compression therapy, we ask that you take some time to read these instructions carefully. Keep them in a safe place so that you can find important information later if needed.

#### 1. Purpose – what are medical compression stockings used for?

Medical compression stockings are an integral part of venous and lymphatic disease prevention and compression therapy. Compression therapy is part of the treatment for venous, lymphatic and mixed edema and is one of the irreplaceable methods of conservative care.

#### 2. Performance characteristics – how do health compression stockings work?

RELAX PREMIUM medical compression stockings have a tested gradient of pressure descent from the ankle upwards. The pressure exerted significantly helps the circulation of blood in the legs.

#### 3. Indications – what symptoms are medical compression stockings used for?

Your RELAX PREMIUM product provides a specific level of compression and helps with prevention and treatment. It is manufactured in the first compression class. It is suitable as prevention of venous and lymphatic diseases or for the following ailments and diseases:

- tired, heavy, aching legs
- scar treatment
- minor varicose veins without edema and venous insufficiency of the C1 grade according to CEAP
- mild swelling of feet, ankles and legs
- prevention of varicose veins in pregnancy
- onset of pregnancy-related varicose veins
- controlled cardiac oedema
- arthritis
- primary or secondary lymphoedema in the stages 0-II according to the International Society of Lymphology (ISL) with no or minimal shape distortion (or lymphoedema in the stage III according to the ISL when higher compression is not tolerated)

RELAX PREMIUM stockings also help to prevent the development of travel thrombosis and are suitable for prolonged sitting and standing, after lymphatic drainage or after cosmetic surgery on dilated skin veins (spider veins).

#### 4. Contraindications – when should medical compression stockings not be worn?

This treatment has no side effects as long as the size chart is followed when choosing the size. Nevertheless, some possible contraindications must be taken into account:

- Wearing stockings is not recommended at night, as the circulation in the veins is already supported by the horizontal position of the body.
- Stockings are not recommended for patients with skin diseases with secretion.
- Stockings are not recommended for patients with allergies to the materials used in their production.

#### 5. Intended target user group

All women who want to have beautiful, healthy and light legs without veins and swelling. All women when standing or sitting for long periods of time (e.g. when travelling). All women who take care of their legs as a preventive measure and women with the indications mentioned above (see point 3).

The intended users also include the patients themselves and doctors, health professionals, nursing staff ensuring the correct application of medical compression stockings.

These can all help you with the correct size selection.

#### 6. Wearing and use time – how long can I wear compression stockings?

Unless your doctor specifies otherwise, wear your compression stockings from morning to evening for an optimal therapeutic effect. If in exceptional cases (e.g. a long-haul flight) the wearing time is significantly exceeded, you have the possibility to keep your compression stockings on.

For the stockings to work optimally, a precisely graduated pressure curve is a prerequisite. Everyday wearing and washing can reduce the necessary therapeutic pressure and elasticity of your compression stockings over time. The recommended usability is therefore 6 months.

#### 7. Instructions for use – what should I know?

- Should you experience pain or excessive skin irritation during wear, remove your stockings immediately and consult your doctor or a healthcare professional.
- Appropriate skin care is essential for compression therapy. In particular, creams or ointments containing fat, but also soap residues, can cause skin irritation and material wear, and thus reduce the effectiveness of medical compression stockings. That's why we offer complementary skin care products that are specially adapted for compression stockings. Ask for advice at specialized points of sale.
- In silicone banded compression stockings, skin irritation may occur in people with sensitive skin (due to a combination of sweat and mechanical stress). To avoid this, it is recommended to slightly move the band several times during the day. Slipping of thigh stockings can also be caused by a large amount of hairs. In similar cases, we recommend the use of a special adhesive for a better grip of the medical compression stockings. If necessary, ask for advice at specialized points of sale.
- Sharp finger and toe nails, defective shoes, jewelry, as well as rough and calloused heels can cause damage to the knit.
- Never repair stockings by yourself or have them repaired by another service provider, as this reduces the quality, safety and efficiency of the medical device. Information on repair, if possible, will be provided to you by our salespeople or our customer service.
- In case of complaints about products such as knit damage, please contact your specialized shops. Only serious events that may lead to a significant deterioration of health or death shall be reported to the manufacturer and the competent state authority. Major events are defined in Article 2 no 65. of Regulation (EU) 2017/745 (MDR).

- A textile label sewn onto the stockings plays an important part in identifying the product. Under no circumstances should you remove this label from your medical compression stocking, cutting it will void your warranty, repair, or replacement.

#### 8. Donning instructions – what should I be careful about when putting on my medical compression stockings?

Always don the product on a dry leg, preferably in the morning before putting pressure on the limb. If you take a shower in the morning and then take care of your skin (see also point 10 – Instructions for care), wait a short time until your skin dries properly. This makes donning much easier. Also prepare your textile or rubber gloves, which are available at specialized points of sale. You can use them later to make it easier to spread the stocking around your leg or to get started at the beginning. The gloves fit particularly well and make donning easier.

When putting the stockings on, make sure your nails are rounded and smooth. Well-kept heels and toenails are also important to prevent damage to the knit. Put on your jewelry or wristwatch only after putting on your stockings. To put your medical compression stockings on comfortably, sit down and place your foot on an elevated surface, such as a small stool.

#### Donning closed-toe medical compression stockings

The donning procedure is illustrated in the front of the instruction manual.

1. Don products with a closed toe by inserting your thumbs inside the stocking and pulling the stocking gradually over your toes and over the heel. The heel is an area where you need to develop more force to pull up the stockings.
2. The highest compression pressure is in the ankle area, the product is narrowest here. Gradually pull up the next part of the stocking by scooping up the knit.
3. Proceed further and pull the stocking towards the knee, or higher, depending on the type of stocking. Finally, smooth out any creases and unevenness in the direction from the ankle with your palms until you feel the tight compression effect of the product over the entire surface of your leg.

#### 9. Storage guidelines – how and how long can I store my medical compression stockings?

The products should be stored in a dry place and protected from direct sunlight and heat.

The hourglass symbol on the box label indicates the latest usability date for the product. This is the maximum shelf life of serial products in the length of 42 months. This period already includes a maximum practical usability of 6 months (see point 6).

#### 10. Care guidelines – what do I have to look out for when caring for my medical compression stockings?

Wash your stockings every day after wearing. We recommend handwashing detergent for gentle care. Alternatively, use a mild laundry detergent (delicate wash at 40° C) without an optical brightener and softener.

#### Tip:

- We recommend turning your compression stockings inside out when washing. Wash them separately or with clothes of the same color.
- A washing bag better protects the stocking material.
- Do not dry clean medical compression stockings or wash them using household detergents.
- After washing, you can roll the stockings into a towel and squeeze. Do not leave wet stockings in a wet towel, air dry them. Under no circumstances should they be dried in the sun or on a radiator. Alternatively, you can delicately tumble dry your medical compression stockings.

Wash up to 40° C (delicates)

Do not bleach

Do not iron

Do not dry clean

Low tumble dry temperatures (delicates)

#### 11. Material composition

RELAX PREMIUM compression stockings contain polyamide and elastane. More detailed information can be found on the textile label sewn on your compression stockings.

**Without latex:** There is no rubber from the milk juice of tropical plants processed in this product.

#### 12. Disposal – how do I dispose of my medical compression stockings?

Please dispose of your medical compression stockings in your normal household waste. There are no special criteria for disposal.

#### 13. Explanation of symbols

Medical Device

Unique Device Identifier

**Single patient – multiple use**  
(see also point 6: Wearing time and usability)

**Non-returnable**

Revision date 09/2022

F A S H I O N   C O L L E C T I O N

*maxis*<sup>®</sup>

**MAXIS a.s.**  
Slezská 2127/13  
120 00 Praha 2, Czech Republic  
[www.maxis-medica.com](http://www.maxis-medica.com)

Made in the Czech Republic