medi

medi  
footsupport  
insoles

medi. I feel better.

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# Instructions

## medi footsupport

medi footsupport insoles are aids that have been developed for feet with special demands.

---

### Intended purpose of foot orthotics

Foot orthotics are medical devices for specific malpositions or poor alignment of the patient's feet.

### Indications

Insoles can be implemented for functional therapy for the following diagnoses:

- **Osteoarthritis of the lower limbs**  
Comfort (pro)
- **Diabetic neuropathy**  
Comfort (pro)
- **Heel spurs / Plantar fasciitis**  
Heel Pain (pro)
- **Hallux rigidus**  
Control (slim)
- **Hallux valgus**  
Business (pro), Business slim (pro), Gentlemen pro, Ballerinas pro, High Heels (pro)
- **Metatarsalgia**  
Control (slim)
- **Pes cavus**  
Comfort (pro)
- **Pes valgus**  
Junior, Active, Business (pro), Business slim (pro), Gentlemen pro, Ballerinas pro
- **Fallen arches**  
Junior, Active, Business (pro), Business slim (pro), Gentlemen pro, Ballerinas pro
- **Flat foot**  
Junior, Active, Business (pro), Business slim (pro), Gentlemen pro, Ballerinas pro
- **Splay foot**  
Business (pro), Business slim (pro), Gentlemen pro, Ballerinas pro

### Contra-indications

None

### Features of the product

- **Cushion**  
The insoles embed the foot so that it is cushioned, protecting it from pressure peaks and relieving load on individual parts of the foot in a targeted manner.
- **Support**  
The supportive function of the insole can relieve load on specific parts of the foot, improve rolling of the foot and helps to optimise the functional stability of the foot.

- **Pain relief**

The structure of the insole and the combination of functional materials can help to reduce the pain.

- **Correction**

The corrective function of the insoles is achieved via various supportive elements positioned at specific points on the foot, e.g. a corrective support for the arch.

---

### **Important Notes**

#### **Instruction for use:**

- Insoles should always be worn as a pair and are only intended for use with intact skin - they cannot be worn on open wounds. Furthermore, the insoles are intended for multiple use by one individual patient. The products must be issued by experts or trained personnel. Furthermore, the product must be used in accordance with the manufacturer's instructions for use (in accordance with the indication, instructions for putting on, storage and care instructions) and the respective orders from the specialist personnel.
- The stress that the insoles are placed under due to general wearing, salves, skin moisture and environmental influences can reduce their service life and impair the medical effectiveness of the insoles. Therefore, we recommend that your doctor and/or specialist retailer checks the insoles regularly.

#### **Intended users:**

- Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

#### **Patient target group:**

- Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

#### **Disposal:**

- You can dispose of this product with your household waste.

#### **Side effects:**

- Sometimes, pressure points can be created or the insole can slip in the shoe. Therefore wear well-fitting shoes, otherwise using the insole will incur risks.
- 

### **Instructions for putting the insoles on**

- The insoles can be individually adjusted to the foot and shoe by an expert.
- If required, remove any insoles that are already in the shoe in order to achieve an optimal fit.
- Some individuals may need to allow their feet to get used to the insoles. This can last several days - up to two weeks.
- If pain and/or discomfort occur whilst wearing the insoles, remove the insoles immediately and contact your doctor or your specialist retailer.
- If you have experienced ongoing problems due to wearing your insoles, please consult your doctor.

---

### **Instructions for care**

- If the insoles need cleaning, clean them with a damp cloth.
  - Do not use any solvents containing acetone to clean them.
  - Do not dry your insoles on the radiator, do not expose them to other direct sources of heat.
- 

### **Information on the warranty, guarantee and liability**

- The product is intended to be used multiple times by one patient and is therefore not intended to be handed over to third parties. If the insole is used to treat more than one person or is altered in an incorrect manner, the product liability will be rendered null and void.
- No claims on the guarantee may be asserted for any damages which are caused by improper usage. Discolouration of the insoles does not constitute the grounds for a complaint.
- The product identification mark on the insole is an important component of the product. If you remove this identification from a medi product, the claim to a guarantee, repair or exchange is rendered null and void.
- In the event of any complaints regarding the product such as damage to the insole or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

### **Transportation and storage conditions**

- The products must be stored in a dry place and out of direct sunlight.
- Insoles have a service life of 42 months. This includes the storage life and the length of time that they may be used for. The length of time that they may be used for is between 6 and 12 months, depending on how much and how heavily they are worn.

The date of the last revision of the instructions is stated on the last page of the instructions. Information on the manufacturer, the conformity symbol and, if required, the authorised representatives, is stated on the packaging.

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