**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**D-95448 Bayreuth**  
**Germany**  
**T +49 921 912-0**  
**F +49 921 912-780**  
**ortho@medi.de**  
**www.medi.de**

medi Australia Pty Ltd  
 83 Fennell Street  
 North Parramatta NSW 2151  
 Australia  
 T +61-2 9890 8696  
 F +61-2 9890 8439  
 sales@mediaustralia.com.au  
 www.mediaustralia.com.au

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43 512 57 95 15  
 F +43 512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium bvba  
 Posthoornstraat 13/1  
 3582 Koersel  
 Belgium  
 T: + 32-11 24 25 60  
 F: +32-11 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

medi Brasil  
 Rua Cristovan de Vita,  
 n.º 260, Un. 30,  
 Centro Logístico Raposo  
 Tavares,  
 Vila Camargo  
 Vargem Grande Paulista - SP  
 CEP 06730-000  
 Brasil  
 T: +55-11-3500 8005  
 sac@medibrasil.com  
 www.medibrasil.com

medi Canada Inc /  
 médi Canada Inc  
 597, Rue Duvernay, Verchères  
 QC Canada J0L 2R0  
 T: +1 450-583-3317 / +1 800 361  
 3153  
 F: +1 888-583-6827  
 service@medicanada.ca  
 www.medicanada.ca

MAXIS a.s.,  
 medi group company  
 Slezská 2127/13  
 120 00 Prague 2  
 Czech Republic  
 T: +420 571 633 510  
 F: +420 571 616 271  
 info@maxis-medica.com  
 www.maxis-medica.com

medi Danmark ApS  
 Vejlegardsvej 59  
 2665 Vallensbaek Strand  
 Denmark  
 T +45-70 25 56 10  
 F +45-70 25 56 20  
 kundeservice@sw.dk  
 www.medidanmark.dk

medi Bayreuth Espana SL  
 C/Canigo 2-6 bajos  
 Hospitalet de Llobregat  
 08901 Barcelona  
 Spain  
 T +34-932 60 04 00  
 F +34-932 60 23 14  
 medi@mediespana.com  
 www.mediespana.com

medi France  
 Z.I. Charles de Gaulle  
 25, rue Henri Farman  
 93297 Tremblay en France Cedex  
 France  
 T +33-1 48 61 76 10  
 F +33-1 49 63 33 05  
 infos@medi-france.com  
 www.medi-france.com

medi Hungary Kft.  
 Bokor u. 21.  
 1037 Budapest  
 Hungary  
 T +36 1 371-0090  
 F +36 1 371-0091  
 info@medi.hu  
 www.medi.hu

medi Nederland BV  
 Heusing 5  
 4817 ZB Breda  
 The Netherlands  
 T +31-76 57 22 555  
 F +31-76 57 22 565  
 info@medi.nl  
 www.medi.nl

medi Polska Sp. z o.o.  
 Zygmunt Starego 26  
 44-100 Gliwice  
 Poland  
 T +48-32 230 60 21  
 F +48-32 202 87 56  
 info@medi-polska.pl  
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda  
 Rua do Centro Cultural, no. 43  
 1700-106 Lisbon  
 Portugal  
 T +351-21 843 71 60  
 F +351-21 847 08 33  
 medi.portugal@medibayreuth.pt  
 www.medi.pt

medi RUS LLC  
 Business Center NEO GEO  
 Butlerova Street 17  
 117342 Moscow  
 Russia  
 T +7-495 374 04 56  
 F +7-495 374 04 56  
 info@medirus.ru  
 www.medirus.ru

## protect.Epico ROM

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### Important notes

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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medi Trading (Shanghai) Co. Ltd.  
Room 105, Building No. 4,  
Lane 500 ShengXia Road  
Pudong New District  
201210 Shanghai  
People's Republic of China  
T: +86-21 61761988  
F: +86-21 61769607  
infocn@medi.cn

Medi Turk Ortopedi Medikal İc ve Dis Tic.  
AS  
Kültür mah. çaldıran sok.14/3  
06420 Çankaya/Ankara  
Turkey  
T +90 312 435 20 26  
F +90 312 434 22 82  
info@medi-turk.com  
www.medi-turk.com

medi UK Ltd.  
Plough Lane  
Hereford HR4 OEL  
Great Britain  
T +44-1432 37 35 00  
F +44-1432 37 35 10  
enquiries@mediuk.co.uk  
www.mediuk.co.uk

medi Ukraine LLC  
Tankova Str. 8, office 35  
Business-center „Flora Park“  
Kiev 04112  
Ukraine  
T +380 44 591 11 63  
F +380 44 392 73 73  
info@medi.ua  
http://medi.ua

medi USA L.P.  
6481 Franz Warner Parkway  
Whitsett, N.C. 27377-3000  
USA  
T +1-336 4 49 44 40  
F +1-888 5 70 45 54  
info@mediusa.com  
www.mediusa.com

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Aluminium, Polyamid, Polyester,  
PU-Schaum

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# protect. Epico ROM

## Intended purpose

protect.Epico ROM is an elbow brace. The product is exclusively to be used for the orthotic fitting of the elbow and only on unbroken skin or if any wounds have been properly covered.

## Indications

- Conservative treatment for dislocation of the elbow with associated injury
- Orthotic care for fractures of the medial or lateral epicondyles
- Postoperative treatment following implantation of prostheses
- Functional aftercare following stable fractures of the distal humerus
- Temporary immobilisation after arthrolysis of the elbow joint

## Contraindications

If the belts are fastened too tightly, this may cause lymphatic congestion. For this reason, the belt bands should be frequently opened.

## Fitting instructions

- Open all straps and put the brace on so that its joint lies directly over the lateral epicondyle.
- First close the two lower arm belts, then the upper arm belts.
- Guide one end of the neck strap through the ring on the arm cuff and fasten the hook and loop strap.
- Pass the other end over your shoulder, guide it through the ring on the hand rest, and fasten the strap.
- The pads can be trimmed to size with scissors if necessary.

## Setting the flexion and extension limits

To set the radius of motion, first press the grey button during flexion and simultaneously shift the joint to the desired number of degrees.

Repeat this process for extension.

Flexion limit: 0° - 120°

Extension limit: 0° - 120°

## Important notes

Adjust the extension and flexion limits on the instructions of your doctor only.

## Care instructions

Hook and loop fasteners should be closed for washing. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the paddings and straps by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

With normal use, the hinge mechanism should not require any maintenance, but it can be lubricated with Teflon spray.

## Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

## Material composition

Aluminum, Polyamide, Polyester, PU foam.

## Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

**Disposal**

The product can be disposed of in the domestic waste.

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